



# Your NovaChat: First Steps

NovaChat 5 · 4<sup>th</sup> Generation  
NovaChat 8 · 4<sup>th</sup> Generation

NovaChat 10 · 7<sup>th</sup> Generation  
NovaChat 12 · 1<sup>st</sup> Generation

## 1 Before You Turn on Your NovaChat® Device

You received a battery charger with your NovaChat device. Connect the charger to the NovaChat device/amplifier. Plug the other end of the charger into a wall outlet.



NovaChat 5



NovaChat 8



NovaChat 10



See  
NovaChat 12  
Detail  
  
NovaChat 12

### NovaChat 12 Detail

NovaChat 12 device  
USB charging port  
(with back of device  
facing up)



If you insert a USB 2.0  
Micro cable (with the  
back of the device facing  
up), the connector will  
insert into the left side of  
the port. It will *not* fill  
the entire port.



If you insert a USB  
3.0 Micro B cable,  
the connector will  
fill the entire port.




## 2 Call for Training

Call a speech and language pathologist or a Saltillo consultant for detailed training and guidance with your device. For a list of consultants in your area, go to <http://saltillo.com/reps>.

## 3 Identify the Buttons on Your Device

You will frequently use the power and volume buttons. The NovaChat 12 includes Menu, Home, and Back buttons.

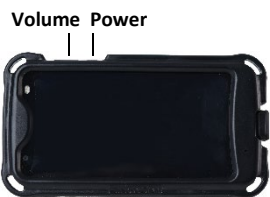


NovaChat 5



NovaChat 8



NovaChat 10

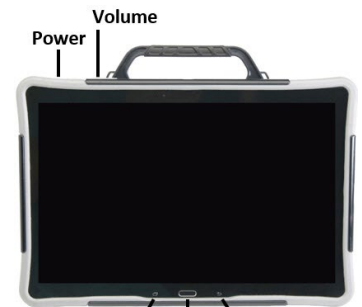


NovaChat 12

## 4 Choose a Vocabulary File

If you want to use a different vocabulary file than the one that is currently active, you can select an alternative. Your speech and language specialist may assist in choosing the best vocabulary.


1. Choose **Menu**.
2. Tap on **Library**.
3. Scroll up and down the list to view the available vocabulary files.
4. Select the file you want.

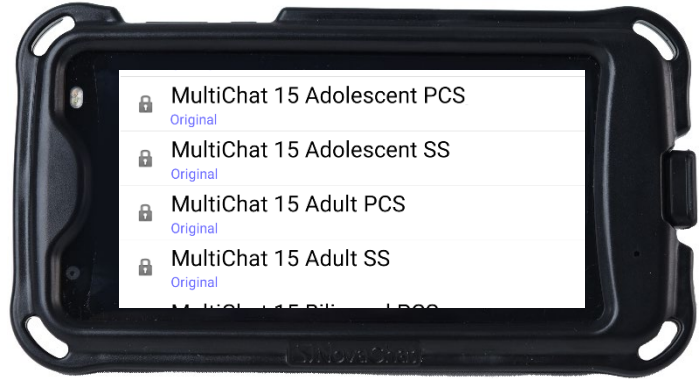


## 5 Choose a Voice

If you want to use a different voice than the one that is currently active, you can choose from a variety of voices.

1. Choose **Menu**.
2. Tap on **Application Settings**.
3. Tap on **Speech Output**.

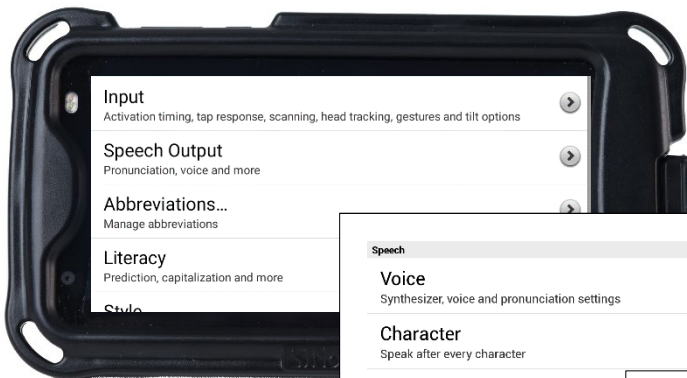



4. Tap on **Voice**.\*

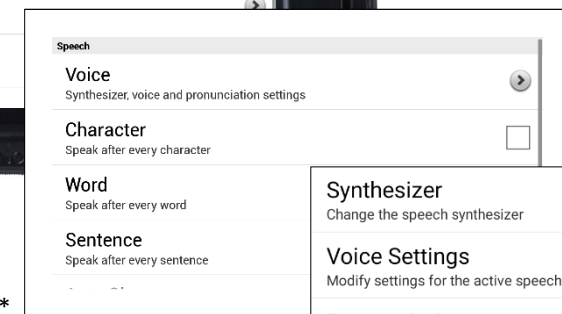

5. Tap on **Voice Settings**.

6. Tap on the arrow on the top far right to open the drop-down menu.
7. Tap on the voice you want.

*\* If you purchased Acapela Synthesizer:*  
After you tap on **Voice** (step 4), tap on **Synthesizer** and select **Acapela**. Then proceed to step 5.

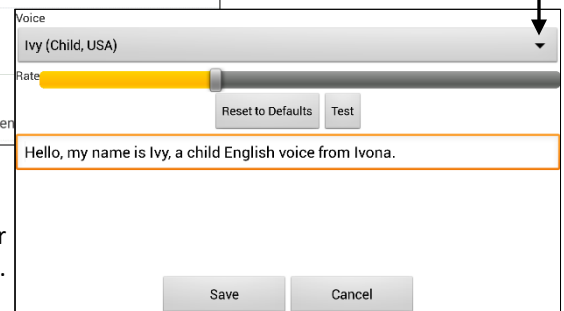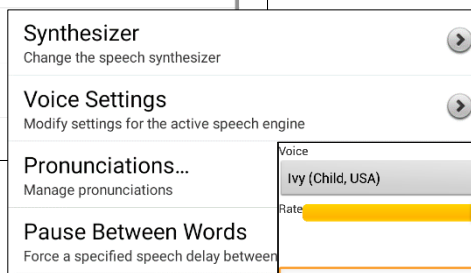

Tap on arrow

8. Tap on **Save**.
9. Select **Back** until you return to the vocabulary page.

## 6 Now, Start Exploring Your Device! Refer to the User's Guide for Assistance.