

Let's teach come


COME is a common verb that we use in daily conversation, whether commenting on something or directing someone.

The word **COME** can mean moving toward something, "**COME** here," traveling to a place, "They have **COME** from Germany," or reaching a specific state or condition, "The screws in the toy are **COMING** loose."




FIND

Find the word in the vocabulary. **COME** is a verb, so it will typically be a green word. It is often located on the home or ACTIONS page of many vocabularies. If you can't find **COME**, use the Word Finder feature to learn where it is. If it is not in the vocabulary, you may want to add it.

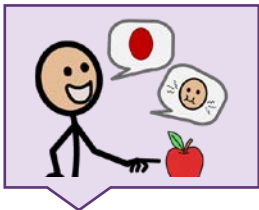

INTRODUCE


Use **COME** in context and try to explain it at the learner's level. Modeling, touching the word in the vocabulary as you say it, may help. Here's an example: "**COME** is when something moves or goes away."

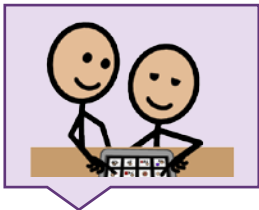

MODEL

Model the word throughout the day, touching **COME** in the vocabulary anytime you say it.

"**COME** here." "This shirt **COMES** in different sizes." "**COME** over for dinner." "Has the package **COME**?" "You start school **COME** September."

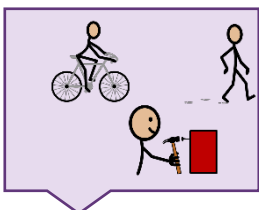


ENCOURAGE

Create activities to provide lots of models and practice with **COME**.

Activity: Travel and talk about where you've **COME** from.

Activity: Dinner invites. "**COME** over this weekend."

Activity: Training your dog to **COME** here.

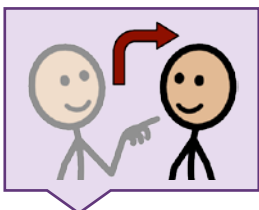

EXPECT

Create opportunities for the learner to say the word. Make sure to wait, and look so they know it is expected.

For requests, "**COME** here."

For questions, "Did it **COME**?"