

let's teach AGAIN

Not **AGAIN!** That's how we feel when our child wants to read the same book, watch the same movie, or eat the same thing for the 100th time.

Repetition is great for learning, so why not take advantage of that? Let's teach the word "**again**" so your child can ask to do it again and again and again!




FIND IT:

Find the word on the device. Use the Word Finder feature to find where the word is located. You might find it under EXTRA WORDS in WordPower or this is a nice place to add the word.




DEFINE IT:

Another time. Once more. Using aided language input, explain it is "do more," "one more time," "over and over."

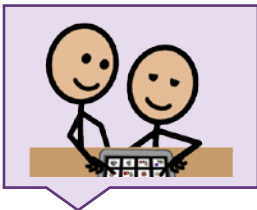


USE IT:

Pay attention to when you already say the word throughout the day. "Come **again**." "Want to do it **again**?" "Let's play **again**." "Go **again**."

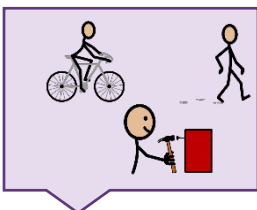

EXPERIENCE IT:

You can do almost any preferred activity again: "Read **again**." "Sing **again**." "Play **again**." "Watch **again**." "Spin **again**."

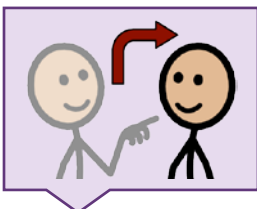

EXPECT IT:

Create opportunities for them to practice using the word. Look and wait expectantly or provide choices "do **again** or all done?"