


How to make the low-tech WordPower 42 Basic PCS book with flips

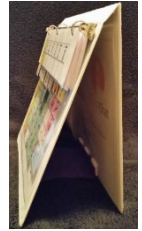

Why use a low-tech option?


- * Use as a **back - up** system for a high- tech system.
- * Use as a **beginner** board. This allows you to transition easily to the high- tech option using the same vocabulary arrangement. Or used while waiting for high-tech device.
- * Use as a **model** for teaching a classroom of students with language deficits.
- * The **communication partner** can also model using the low - tech option.
- * Use during **water** activities. If you punch holes in it, probably not great for submerging, but would last a decent amount of time to water splashes (water table). If you wanted to make the hole punched sheets more waterproof, leave enough lamination above the sheets to punch holes.










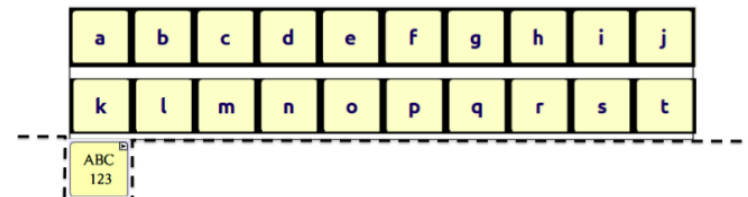



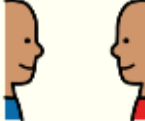














What materials are recommended?











- ✓ **PDF file** of the book
- ✓ **Printer**-preferably color
- ✓ **Lamination** – For the book pictured here, I used a home laminator with 3 ml heat sealed sheets. I have used this laminator for years (over 10) for many different things (placemats, swimming games charts, etc.)
- ✓ **Binding** – I used 3 individual rings and also a binder. I chose to leave the binder as is and added a strap to the outside of the book to give it a base (attached with Velcro). This allowed the flips to stand up and then were easily flipped down instead of having to flip from the back. You could select any binding option that suits you. Basically you will want the flips to be able to flip up.
- ✓ **Hole punch**- a 3 hole punch works nicely. This will be determined by the method of binding.
- ✓ **Scissors**










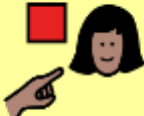


How?




1. **Print** out PDF file.
2. **Add any custom** pictures or words to appropriate flip locations. You can quickly glue a small square on a blank or over a word you do not find necessary for your client. It will be laminated over and kept secure.
3. **Cut** out the pieces. The full page vocabulary should sit at the correct position to align the flips the way they are placed on the pages. You will see a dotted line between the flips to guide you in cutting. This cutting guide will provide you with the top of the next flip. Cut around the flip tabs and follow the line over, as well as straight across the bottom of the other flips. This should align the tops of the flips and leave equal space for the binding option. Printers vary in the way they align and print so check to see that it aligns correctly when you begin cutting.
4. **Laminate**- If you are using a page lamination system, you will simply put the pieces in the pages allowing enough space to leave a laminated edge. If you are using a roller type of laminator, make sure you leave enough space around the edges for the cut and a laminated edge.
5. **Cut** the pieces again- leave a small amount a lamination around edges to keep a seal (lasts longer this way)
6. **Bind**-Use a 3-hole punch or other punch option. (Rings or binder) Check to see the hole alignment matches as you go. Or use a binding machine.
7. **Other** -To make the binder stand up- you can add a strap to back of the folder with Velcro.














PEOPLE 	QUESTN ?	PLACES 	SOCIAL 	TIME 	GROUPS 	ABC 123
I	can	to	ACTIONS 	come 	good 	DESCRB 
my 	do	drink 	eat 	feel 	a	more 
it 	have	finish 	help 	go 	the	that 
you 	is	like 	listen 	play 	stop 	EXTRA And It The WORDS Up
your 	don't X	.	want 	watch 	work 	clear


me 	mom 	dad 	sister 	brother 	grandma 	grandpa 
she 	he 					
PEOPLE 						

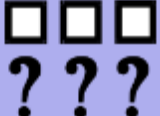
her 	his 	we 	they 	girl 	boy 	baby 
people 	family 	friend 	children 			child 

SCHOOL PEOPLE 						teacher 
						student 


what 	when 	where 	how 	who 	why 	which one 
?	question 	What will we do... 	what time... 			
	QUESTN 					



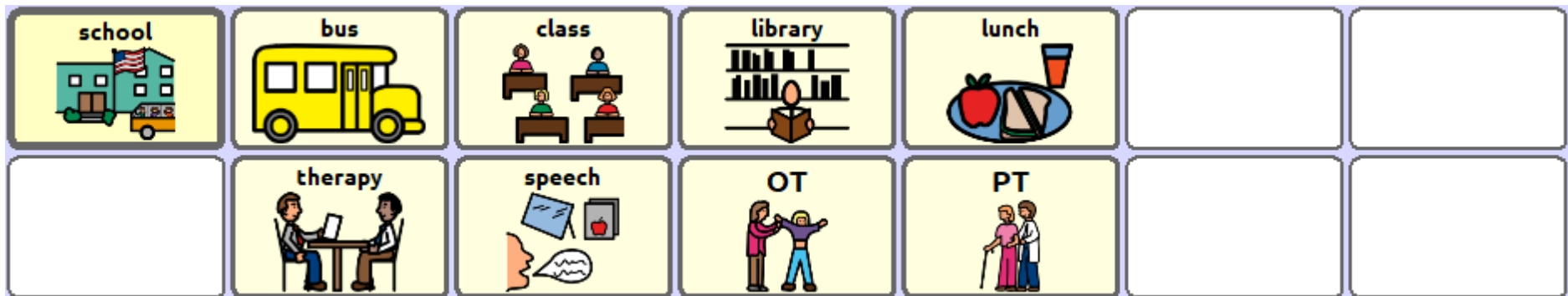


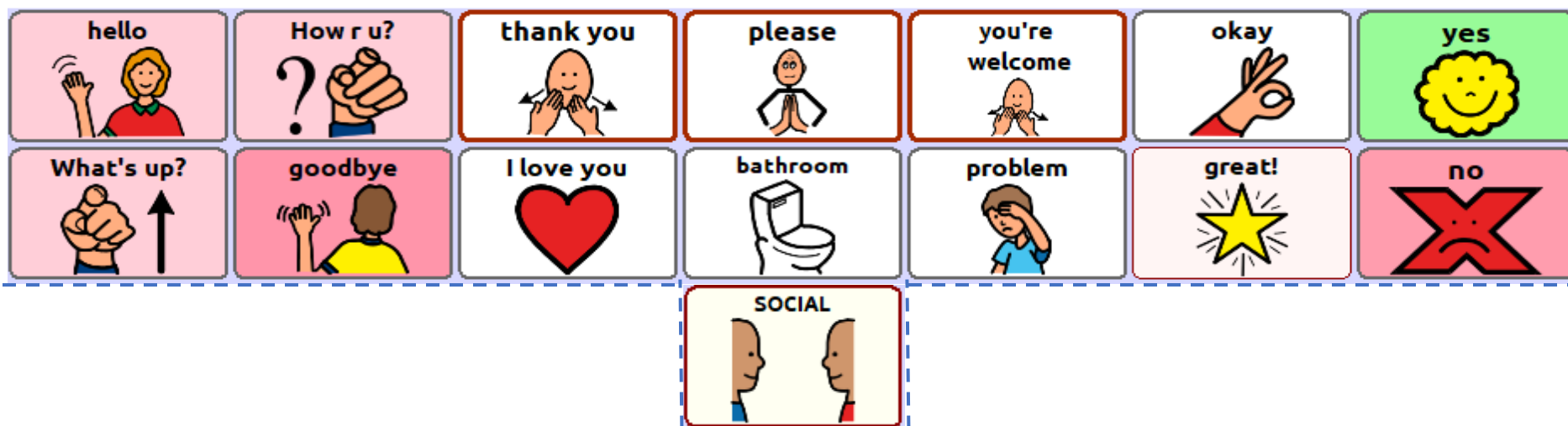

are	bathe 	brush 	buy 	call 	get 	give 
could	clean 	cook 	draw 	drive 	hurt 	know 

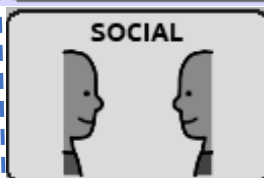



did	jump 	look 	open 	put 	make 	need 
will	would	ride 	shop 	sit 	read 	see 

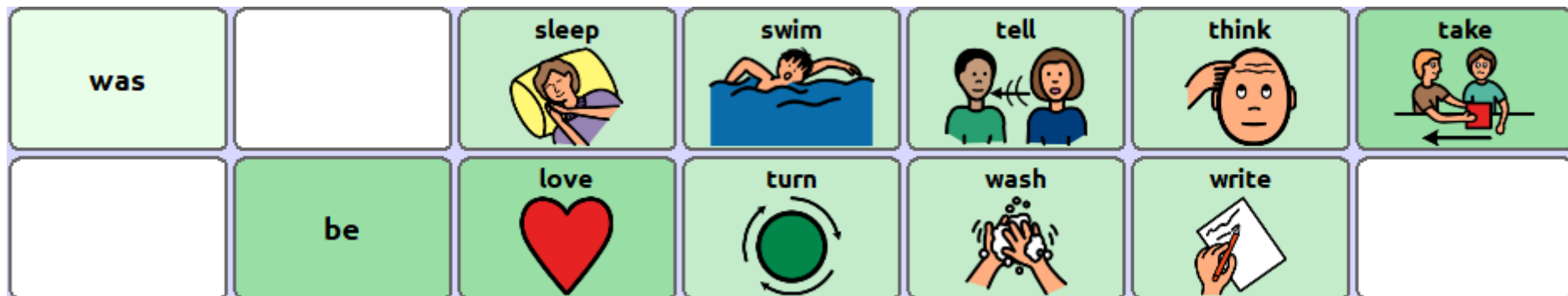







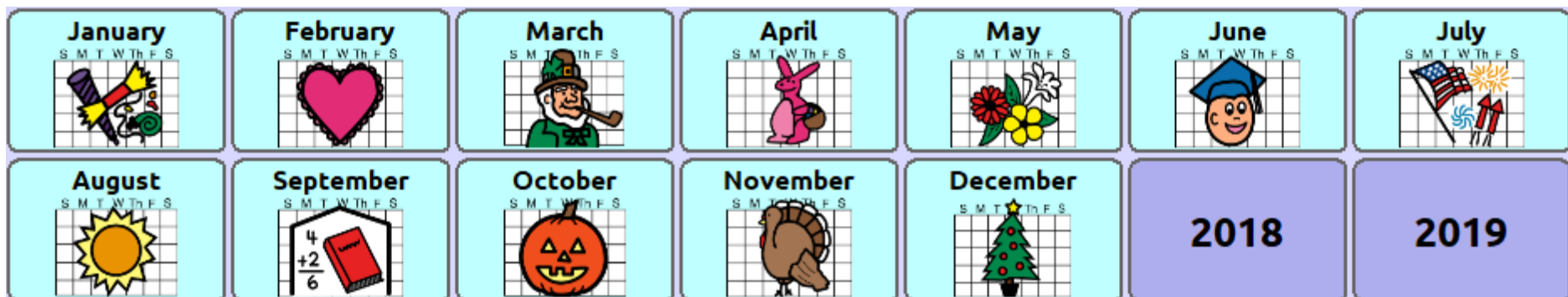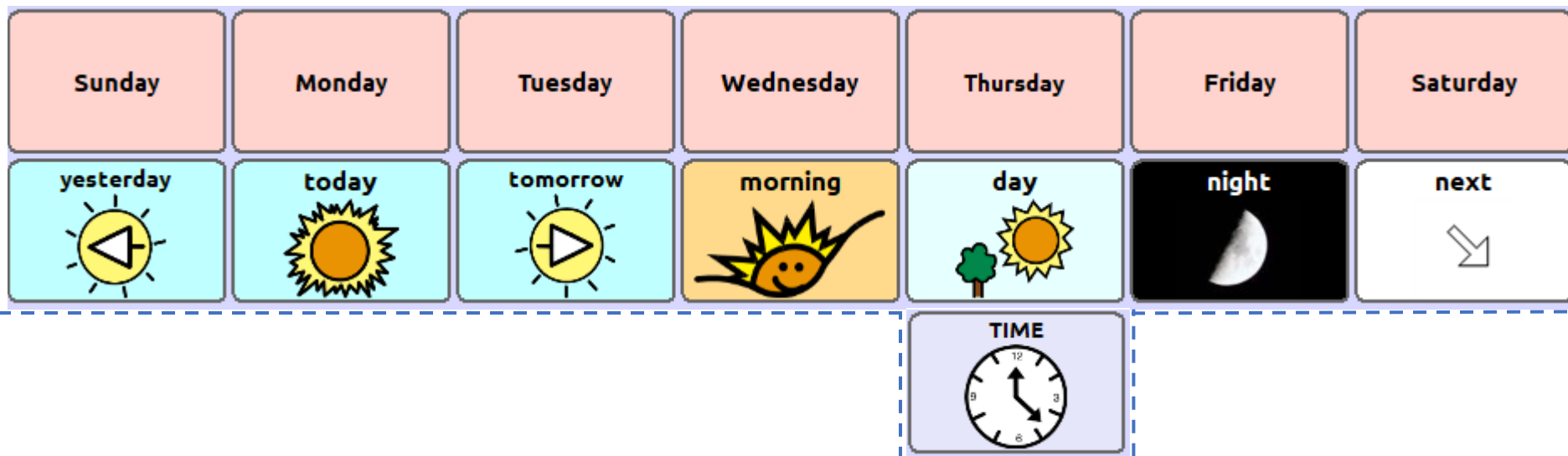




















1
3




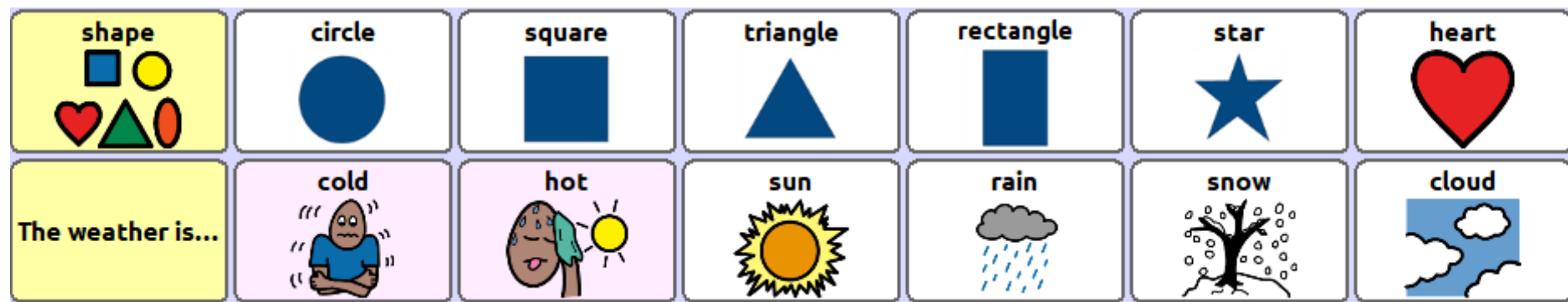
1
4



season 	spring 	summer 	fall 	winter 	soon 	until 
early 	late 	before 	after 	now 	later 	


DRINKS 	water 	juice 	milk 	choc milk 	smoothie 	shake 
SNACKS 	crackers 	cookies 	fruit snack 	pudding 	yogurt 	pretzels 

GROUPS






2
2

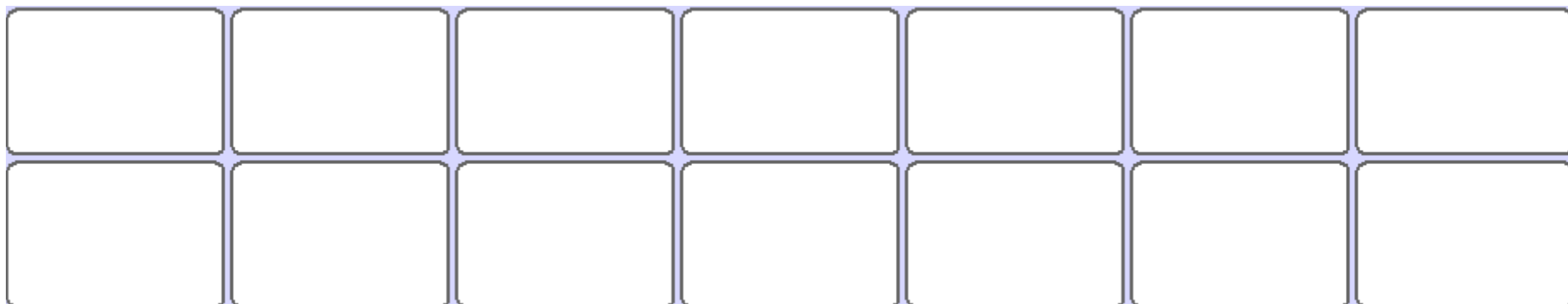

2
3

A

B

C

D

E

F

G

H

I

J

K

L

M

N

ABC
123

O

P

Q

R

S

T

U




V




W


X

Y

Z

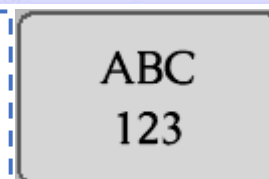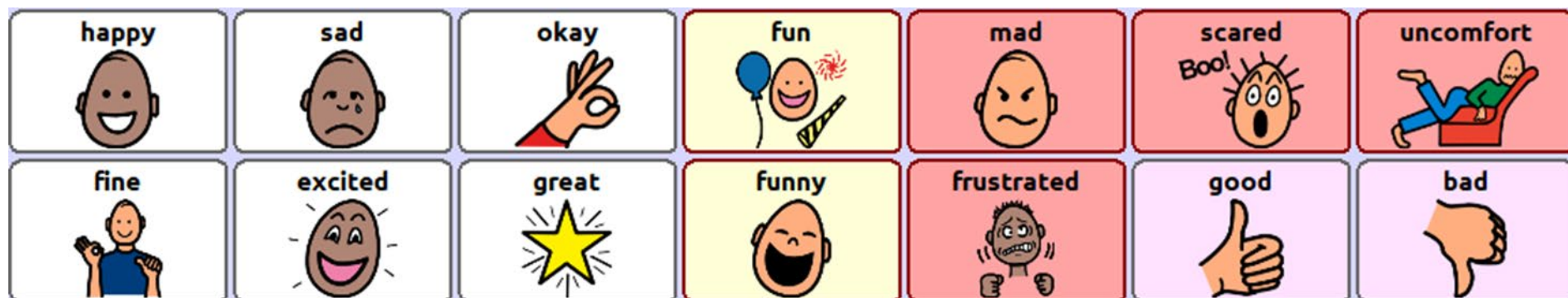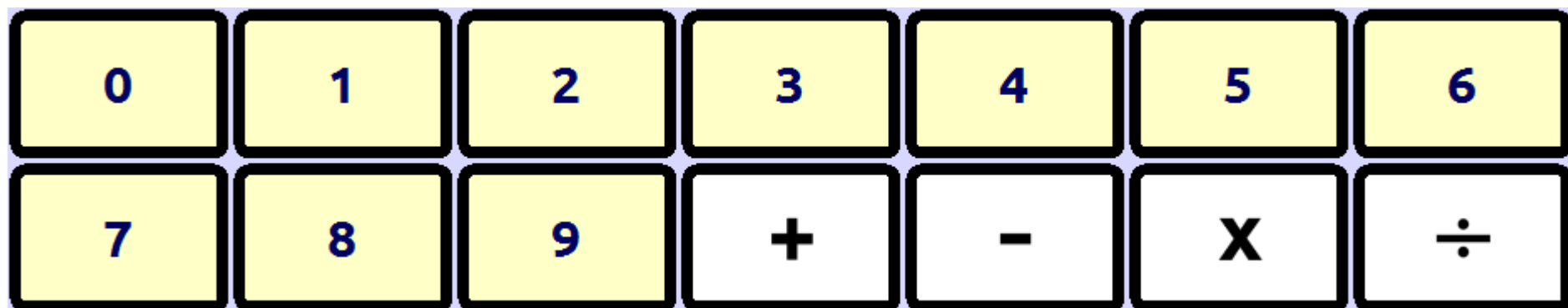

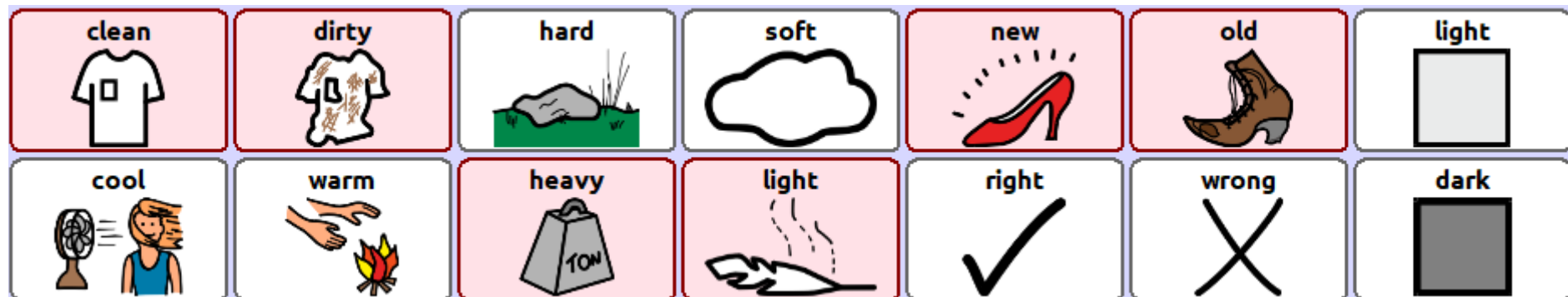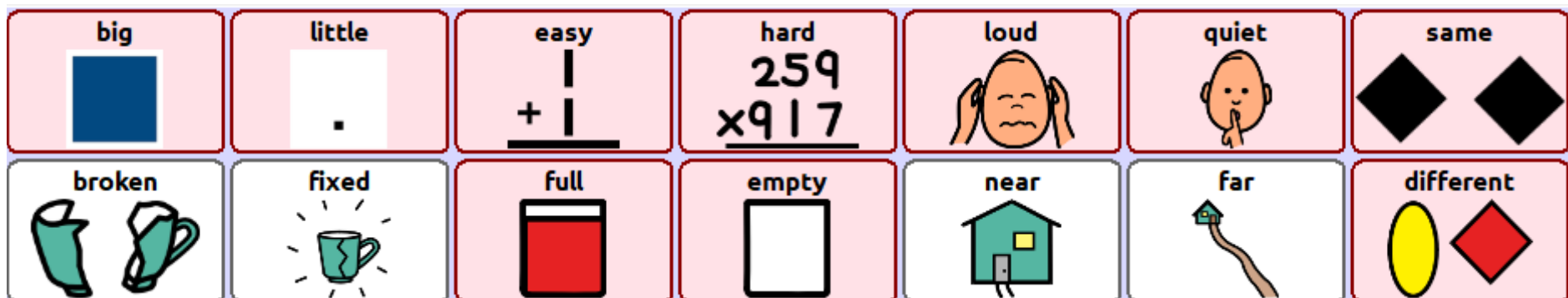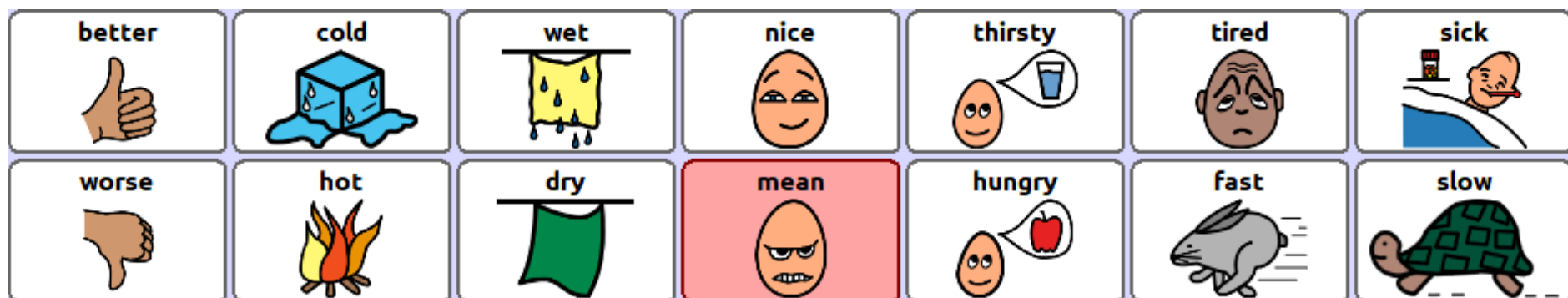


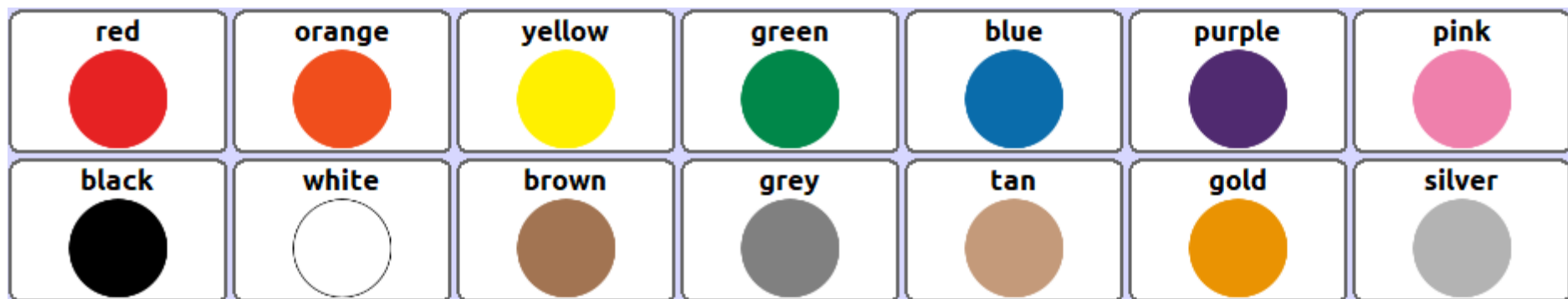



3
2


cut this area out-keep strip on right side-this will lead to the flip for 'extra words'

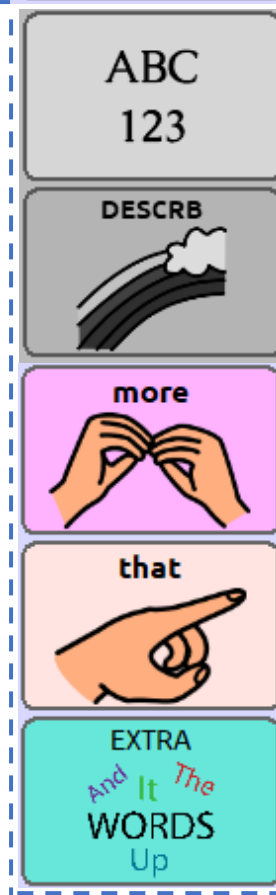

nothing



this

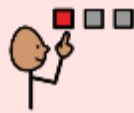

but



all



up



with

not



here



there



left



down



right

