


## How to make the low-tech WordPower60 PCS book with flips

### Why use a low-tech option?

- \* Use as a **back - up** system for a high- tech system.
- \* Use as a **beginner** board. This allows you to transition easily to the high-tech option using the same vocabulary arrangement. Or used while waiting for high-tech device.
- \* Use as a **model** for teaching a classroom of students with language deficits.
- \* The **communication partner** can also model using the low - tech option.
- \* Use during **water** activities. If you punch holes in it, probably not great for submerging, but would last a decent amount of time to water splashes (water table). If you wanted to make the hole punched sheets more waterproof, leave enough lamination above the sheets to punch holes.

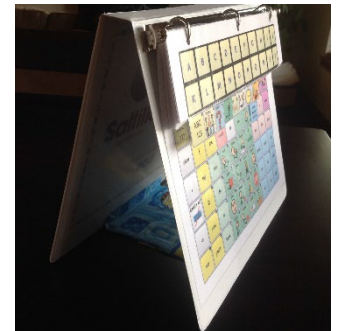





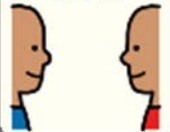






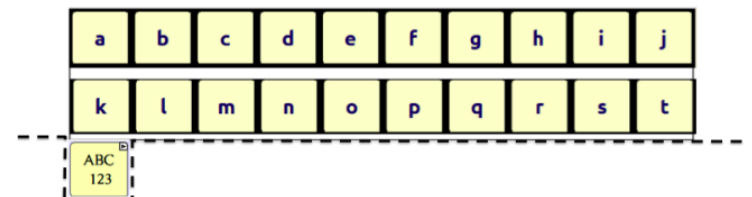
















### What materials are recommended?

- ✓ **PDF file** of the book
- ✓ **Printer**-preferably color
- ✓ **Lamination** – For the book pictured here, I used a home laminator with 3 ml heat sealed sheets. I have used this laminator for years (over 10) for many different things (placemats, swimming games charts, etc.)
- ✓ **Binding** – Use 3 individual rings or a binder. I chose to leave the binder as is and added a felt stand to the outside of the book (attached with Velcro). This allowed the flips to stand up and then were easily flipped down instead of having to flip from the back. You could select any binding option that suits you. Basically, you will want the flips to be able to flip up
- ✓ **Hole punch**- a 3-hole punch works nicely. This will be determined by the method of binding.
- ✓ **Scissors**

### How?

1. **Print** out PDF file.
2. **Add** any **custom** pictures or words to appropriate flip locations. You can quickly glue a small square on a blank or over a word you do not find necessary for your client. It will be laminated over and kept secure.
3. **Cut** out the pieces. The full-page vocabulary should sit at the correct position to align the flips the way they are placed on the pages. You will see a dotted line between the flips to guide you in cutting. This cutting guide will provide you with the top of the next flip. Cut around the flip tabs and follow the line over, as well as straight across the bottom of the other flips. This should align the tops of the flips and leave equal space for the binding option. Printers vary in the way they align and print so check to see that it aligns correctly when you begin cutting.
4. **Laminate**- If you are using a page lamination system, you will simply put the pieces in the pages allowing enough space to leave a laminated edge. If you are using a roller type of laminator, make sure you leave enough space around the edges for the cut and a laminated edge.
5. **Cut** the pieces again- leave a small amount a lamination around edges to keep a seal (lasts longer this way)
6. **Bind**-Use a 3-hole punch or other punch option. (Rings or binder) Check to see the hole alignment matches as you go. Or use a binding machine.
7. **Other** -To make the binder stand up- you can add a strap or piece of material to back and front of the folder with Velcro.



ABC 123	PEOPLE 	QUESTN ?	ACTIONS 	PLACES 	SOCIAL 	TIME 	GROUPS 	DESCRB 	good 
clear	I	me	don't	to	be-	come 	WORDS And It The WORDS Up	a	more 
.	my	can	eat 	drink 	finish 	go 	this	the	that
delete wd 	it	do	get 	help 	know 	listen 	about	and	at
-s	you	is	like 	need 	play 	stop 	for	in	with
we	your	have	think 	want 	watch 	work 	of	on	out

A	B	C	D	E	F	G	H	I	J
K	L	M	N	O	P	Q	R	S	T




1

ABC  
123




U	V	W	X	Y	Z		?	.	!
😊	😞	😎	🎉	🎵	Space		clear		

2



SCHOOL PEOPLE 								student 	teacher 


5

what	when	where	whose	how	who	why	which	?	question 
would	could	should	will	does	Guess what?	What will we do... 	what time... 		


6





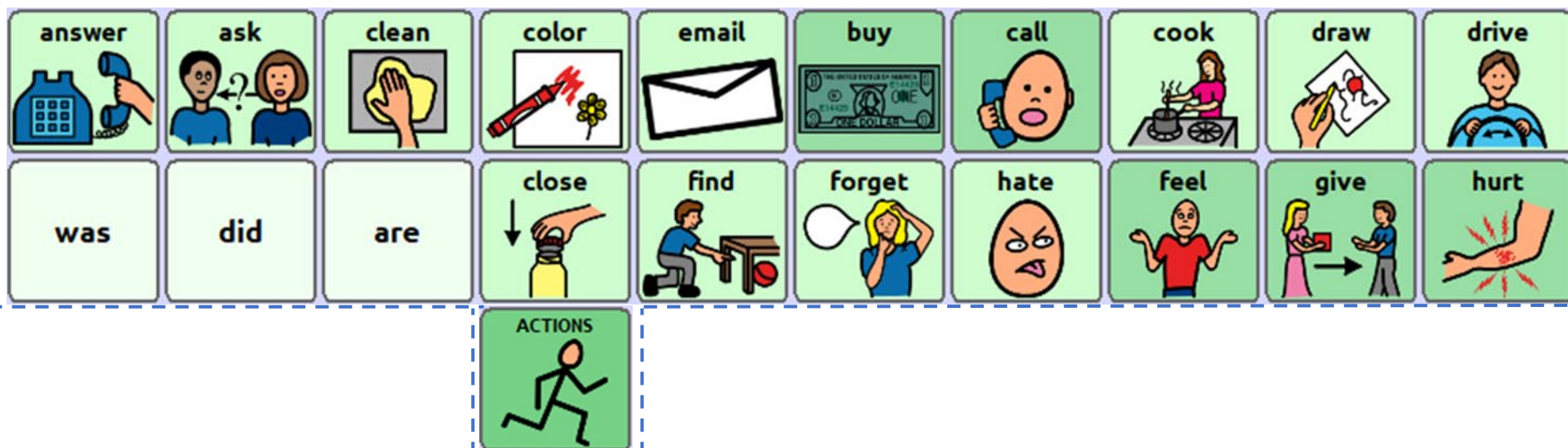





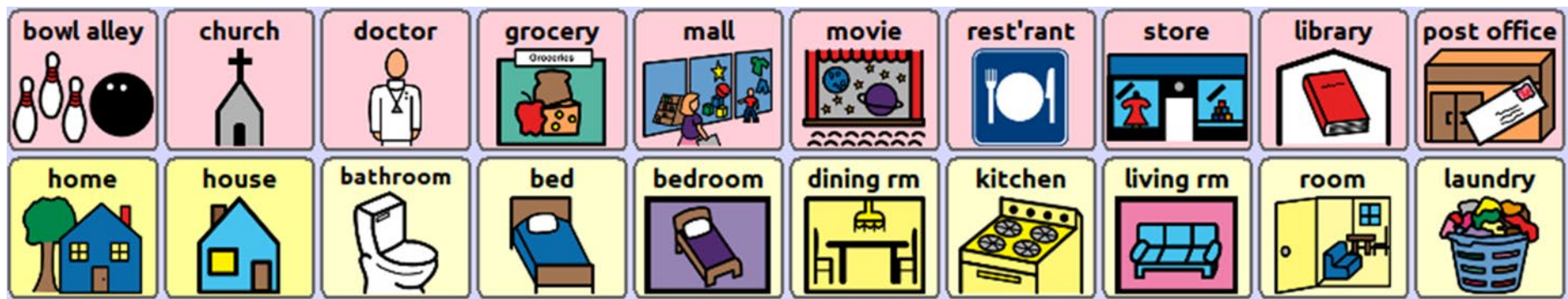







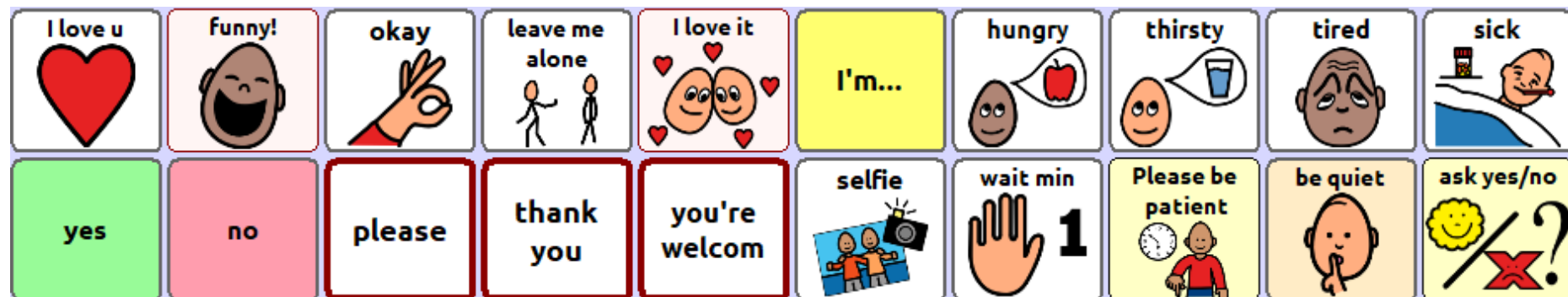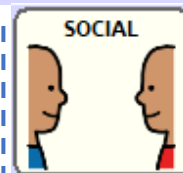










1  
3

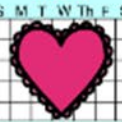


1  
4

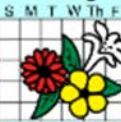
<b>PERSONAL &amp; QUESTIONS</b> 	<b>My name</b> 	<b>I live at</b> 	<b>My b'day</b> 	<b>I am _ years old</b>	<b>school</b> 	<b>I like to</b> 	<b>My pets</b> 	<b>phone #</b> 	
<b>How about you?</b>		<b>name?</b> 	<b>live?</b> 	<b>birthday?</b> 	<b>How old are you?</b>	<b>school?</b> 	<b>like to do?</b> 	<b>pets?</b> 	<b>phone #?</b> 

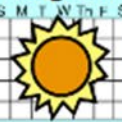1  
5


<b>Sun</b>	<b>Mon</b>	<b>Tues</b>	<b>Wed</b>	<b>Thur</b>	<b>Fri</b>	<b>Sat</b>	<b>yest'day</b> 	<b>today</b> 	<b>tom'row</b> 
<b>Jan</b> 	<b>Feb</b> 	<b>March</b> 	<b>April</b> 	<b>May</b> 	<b>June</b> 	<b>July</b> 	<b>August</b> 	<b>Sept</b> 	<b>Oct</b> 

**TIME**  


1  
6

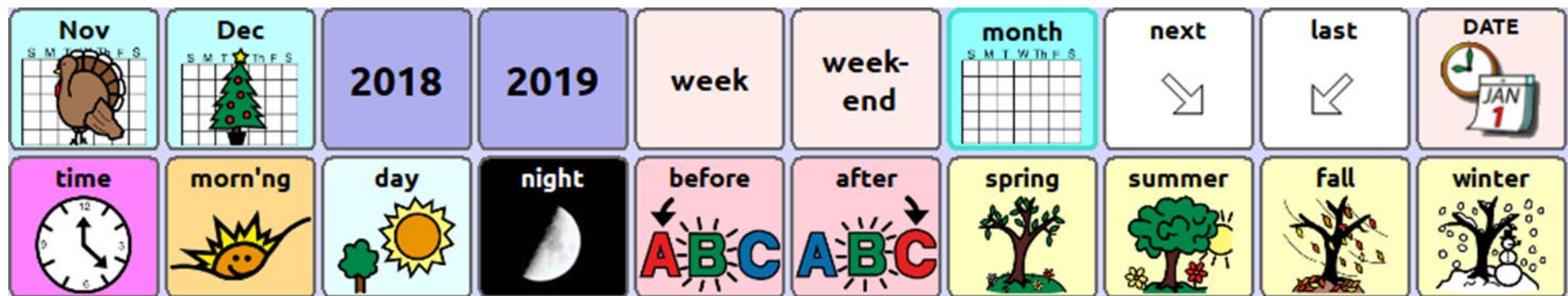


1  
7

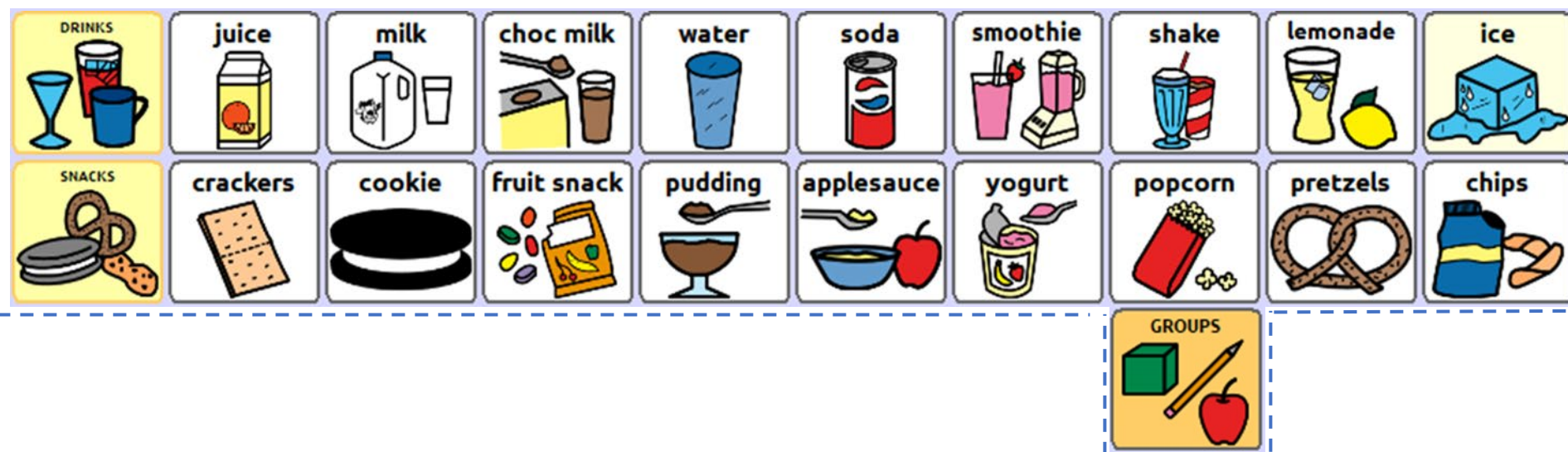


1  
8




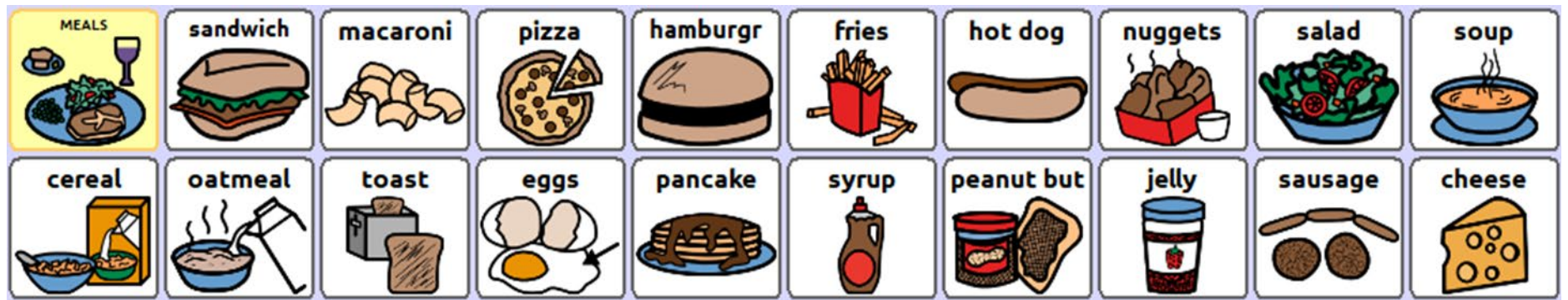

1  
9

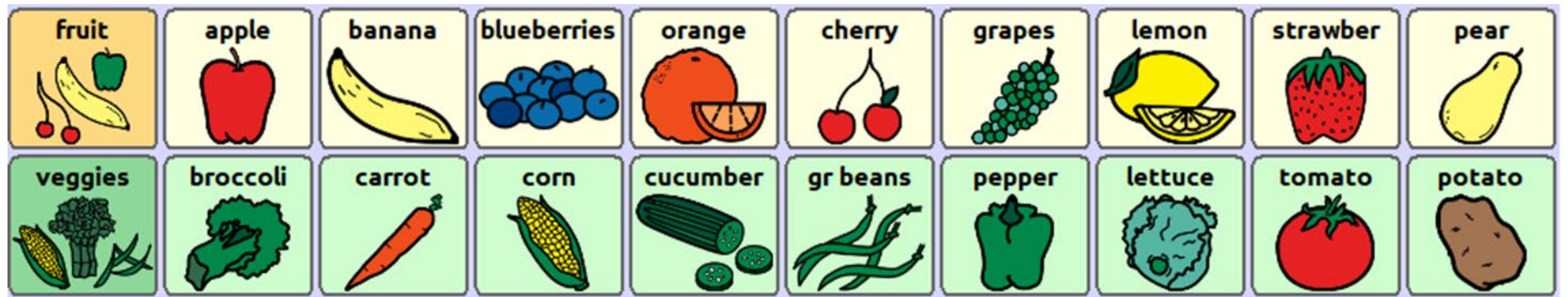


2  
0

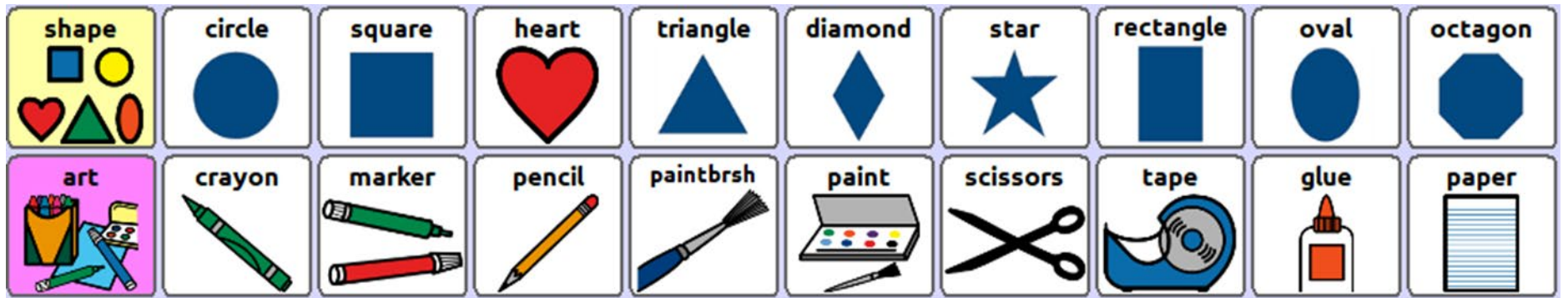

2  
1




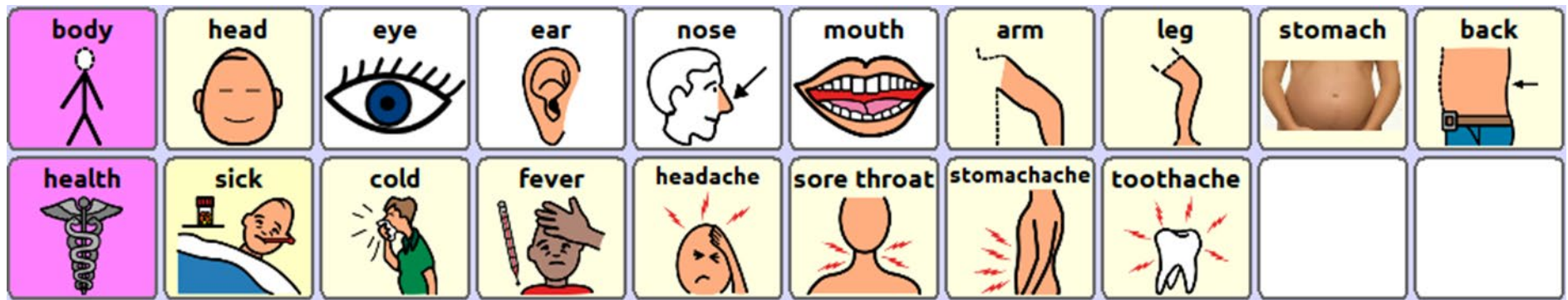

2  
2



2  
3

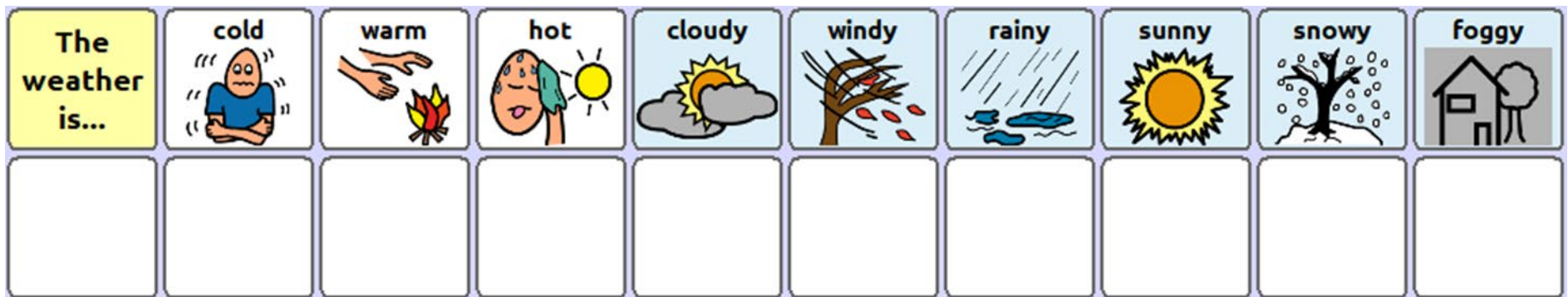




2  
4



2  
5

not	here	there	top	middle	bottom	up 	down 	left 	right 
off	under	over	through	behind	around	again	all	away	be- tween

2  
6

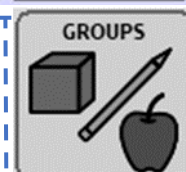

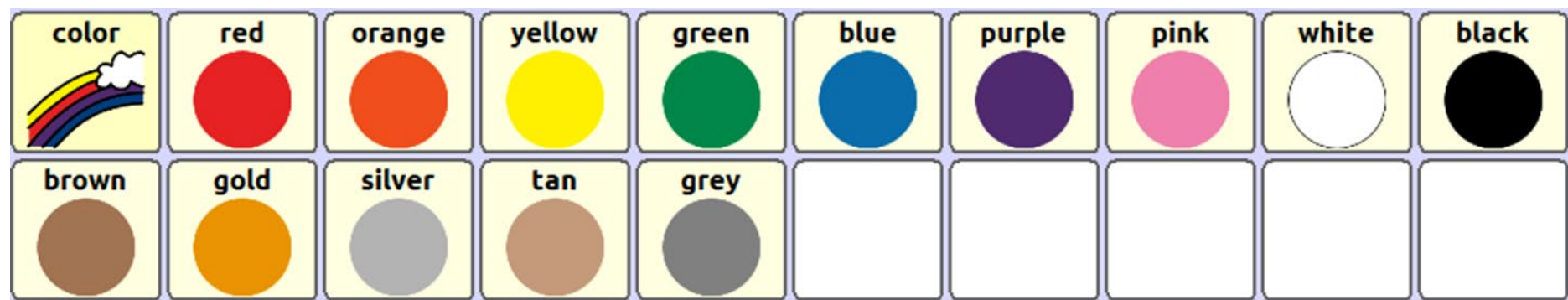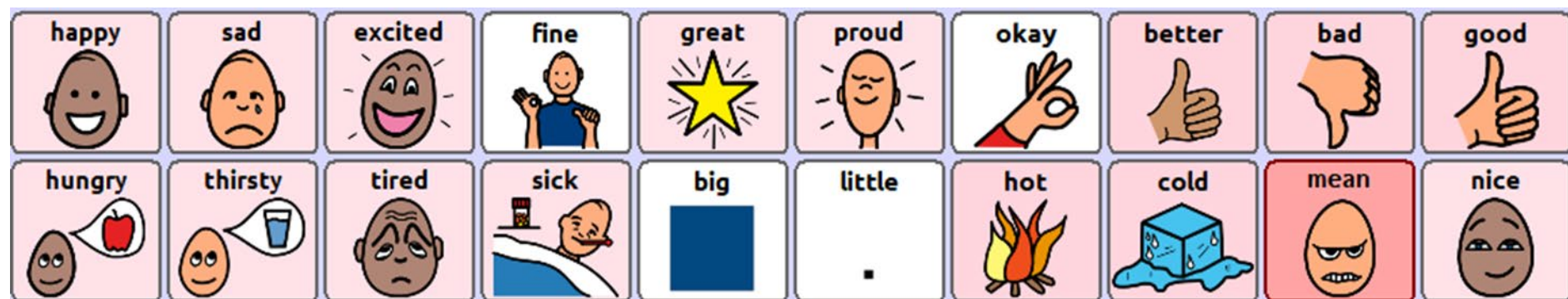

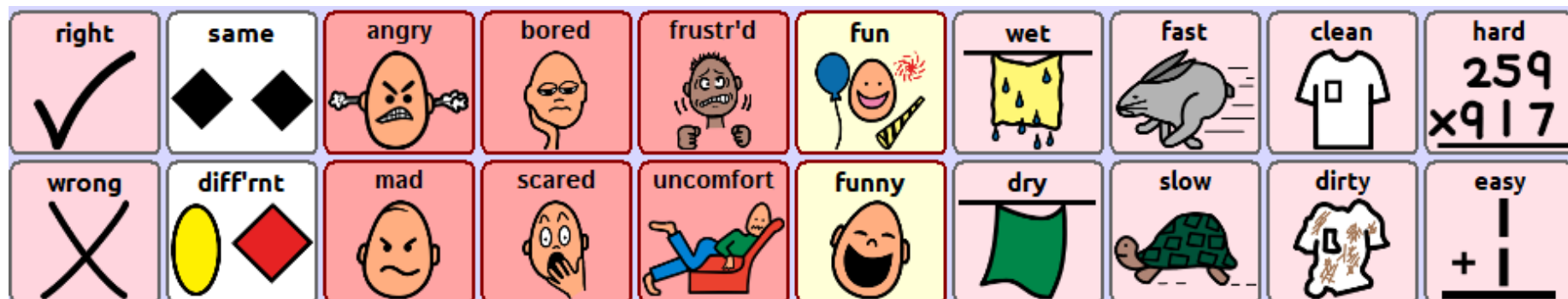

2  
9



3  
0