


## How to make the low-tech WordPower 60 Basic SS book with flips

### Why use a low-tech option?

- \* Use as a **back - up** system for a high- tech system.
- \* Use as a **beginner** board. This allows you to transition easily to the high- tech option using the same vocabulary arrangement. Or used while waiting for high- tech device.
- \* Use as a **model** for teaching a classroom of students with language deficits.
- \* The **communication partner** can also model using the low - tech option.
- \* Use during **water** activities. If you punch holes in it, probably not great for submerging, but would last a decent amount of time to water splashes (water table). If you wanted to make the hole punched sheets more waterproof, leave enough lamination above the sheets to punch holes.

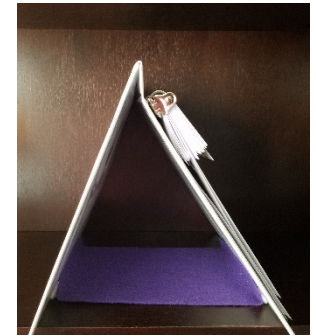










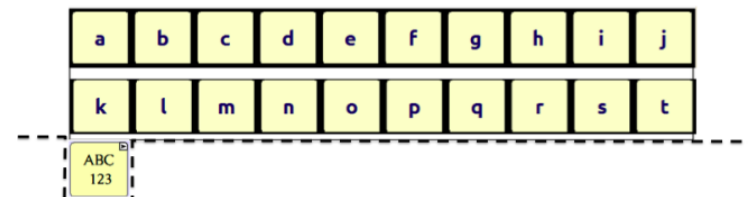






































### What materials are recommended?

- ✓ **PDF file** of the book
- ✓ **Printer**-preferably color
- ✓ **Lamination** – For the book pictured here, I used a home laminator with 3 ml heat sealed sheets. I have used this laminator for years (over 10) for many different things (placemats, swimming games charts, etc.)
- ✓ **Binding** – Use 3 individual rings or a binder. I chose to leave the binder as is and added a felt stand to the outside of the book (attached with Velcro). This allowed the flips to stand up and then were easily flipped down instead of having to flip from the back. You could select any binding option that suits you. Basically you will want the flips to be able to flip up
- ✓ **Hole punch**- a 3 hole punch works nicely. This will be determined by the method of binding.
- ✓ **Scissors**

### How?

1. **Print** out PDF file.
2. **Add** any **custom** pictures or words to appropriate flip locations. You can quickly glue a small square on a blank or over a word you do not find necessary for your client. It will be laminated over and kept secure.
3. **Cut** out the pieces. The full page vocabulary should sit at the correct position to align the flips the way they are placed on the pages. You will see a dotted line between the flips to guide you in cutting. This cutting guide will provide you with the top of the next flip. Cut around the flip tabs and follow the line over, as well as straight across the bottom of the other flips. This should align the tops of the flips and leave equal space for the binding option. Printers vary in the way they align and print so check to see that it aligns correctly when you begin cutting.
4. **Laminate**- If you are using a page lamination system, you will simply put the pieces in the pages allowing enough space to leave a laminated edge. If you are using a roller type of laminator, make sure you leave enough space around the edges for the cut and a laminated edge.
5. **Cut** the pieces again- leave a small amount a lamination around edges to keep a seal (lasts longer this way)
6. **Bind**-Use a 3-hole punch or other punch option. (Rings or binder) Check to see the hole alignment matches as you go. Or use a binding machine.
7. **Other** -To make the binder stand up- you can add a strap or piece of material to back and front of the folder with Velcro.



ABC 123	PEOPLE 	QUESTN 	ACTIONS 	SOCIAL 	PLACES 	TIME 	GROUPS 	DESCRB 	good 
clear	I 	me 	to	come 	that 	a-	the	and 	more 
.	my 	is	eat 	drink 	finish 	get 	all 	at	COLORS 
EXTRA And It The WORDS Up	it 	can	go 	help 	open 	put 	in 	for	on 
here 	you 	do	like 	play 	read 	stop 	out 	up 	off 
yes 	your 	no 	want 	take 	tell 	turn 	watch 	down 	with






1	2	3	4	5	6	7	8	9	0
+	-	X	÷	=	first 	next 	second 	third 	last 










3

	people 	friend 	family 	mom 	dad 	grandma 	grandpa 	sister 	brother 
they 	mine 	we 		she 	he 			girl 	boy 
PEOPLE 									

4

<b>SCHOOL PEOPLE</b> 								<b>student</b> 	<b>teacher</b> 


5

<b>what</b> 	<b>when</b> 	<b>where</b> 	<b>who</b> 	<b>why</b> 	<b>how</b> 	<b>which</b> 		<b>?</b>	<b>question</b> 
<b>would</b>	<b>could</b>	<b>will</b>	<b>does</b>	<b>can</b>		<b>Guess what?</b>		<b>What will we do...</b> 	


6

<b>QUESTN</b> 
---























would	are	be	find 	jump 	learn 	leave 	love 	make 	need 
will	could	line up 	pull 	push 	rememb 	ride 	say 	see 	show 

9

did		run 	shop 	sing 	sit 	sleep 	talk 	walk 	
think 			stand 	swim 	swing 	write 	wash 	wear 	work 

10

blow 	blush 	bowl 	brush 	catch 	chew 	clap 	cry 	dress up 	dry 
email 	fall 	fly 	forget 	guess 	hate 	hope 	kick 	kiss 	live 

11













12



13












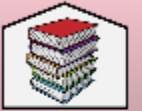













hungry 	thirsty 	tired 	sick 	be quiet 	cool! 	Great! 	uh oh.. 	No way! 	I love u 












14

PERSONAL 	My name 	I live at 	My b'day 	I am _ years old	school 	I like to 	My pets 	How about you?

15


















<b>airport</b> 	<b>bowl alley</b> 	<b>church</b> 	<b>doctor</b> 	<b>grocery</b> 	<b>mall</b> 	<b>movie</b> 	<b>rest'rant</b> 	<b>store</b> 	<b>library</b> 
<b>house</b> 	<b>bathroom</b> 	<b>bedroom</b> 	<b>closet</b> 	<b>dining rm</b> 	<b>kitchen</b> 	<b>living rm</b> 	<b>laundry</b> 	<b>door</b> 	<b>window</b> 
<b>PLACES</b> 									

<b>outside</b> 	<b>beach</b> 	<b>camp</b> 	<b>farm</b> 	<b>lake</b> 	<b>ocean</b> 	<b>park</b> 	<b>playgrnd</b> 	<b>pool</b> 	<b>zoo</b> 
<b>school</b> 	<b>class</b> 	<b>bus</b> 	<b>library</b> 	<b>lunch</b> 	<b>therapy</b> 	<b>speech</b> 	<b>OT</b>	<b>PT</b>	


Sun	Mon	Tues	Wed	Thur	Fri	Sat	yest'day 	today 	tom'row 
time 	morn'ng 	day 	night 	week	week-end	before 	after 	now 	later 

TIME




month 	Jan 	Feb 	March 	April 	May 	June 	July 	August 	Sept 
Oct 	Nov 	Dec 	2017	2018		spring 	summer 	fall 	winter 




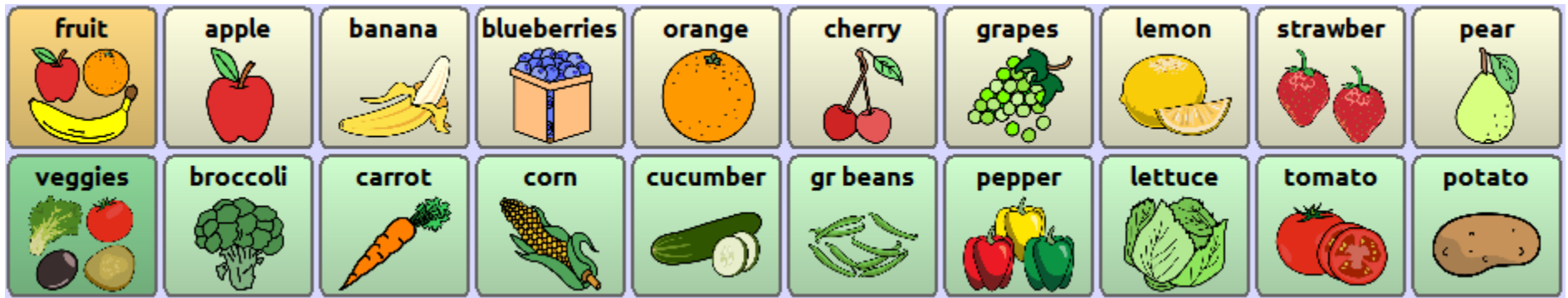

22

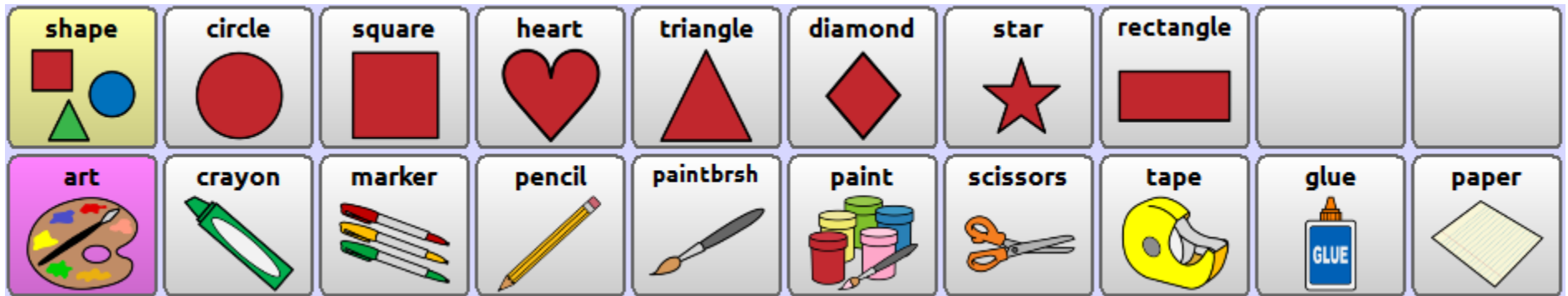


23

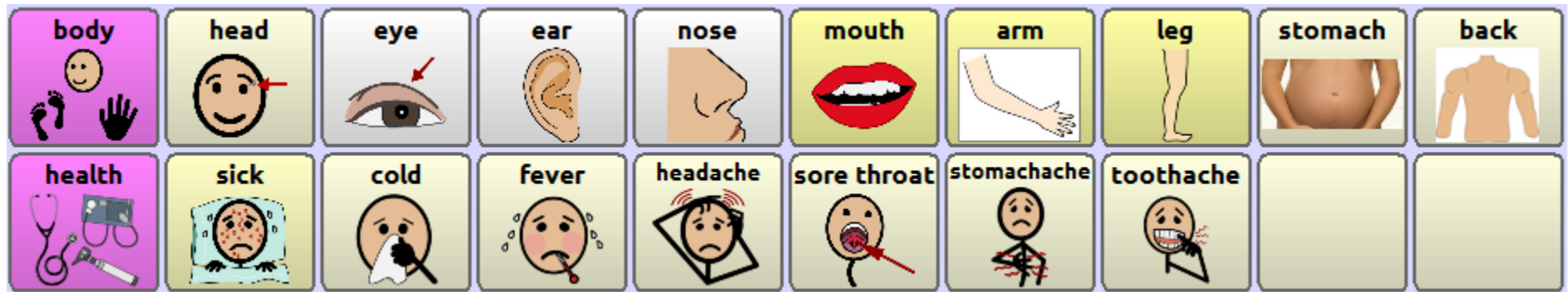


24



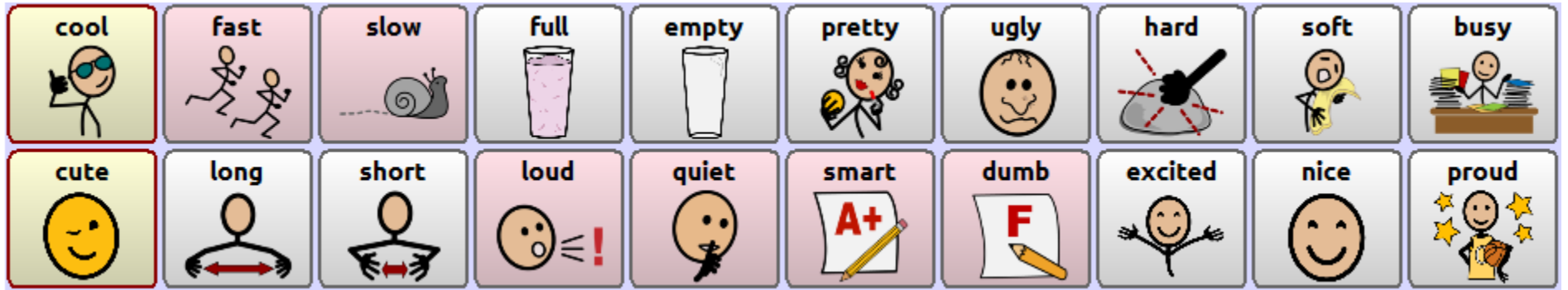




29

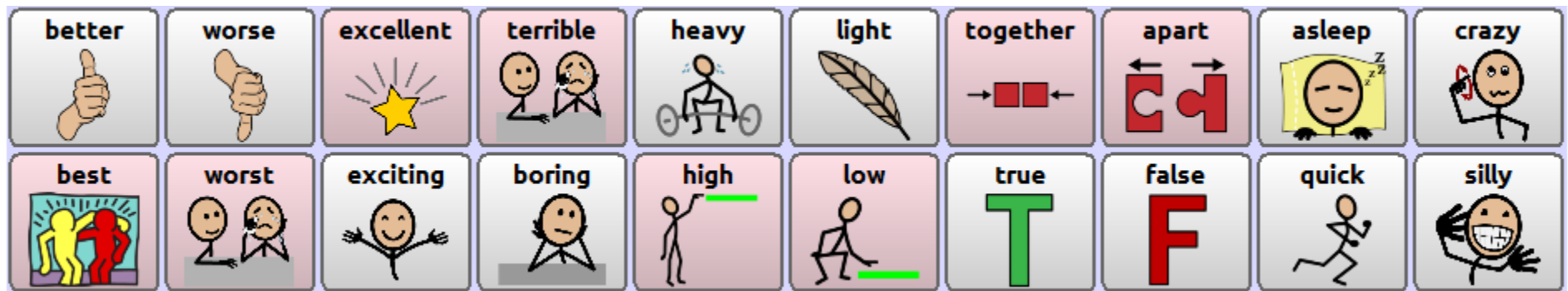



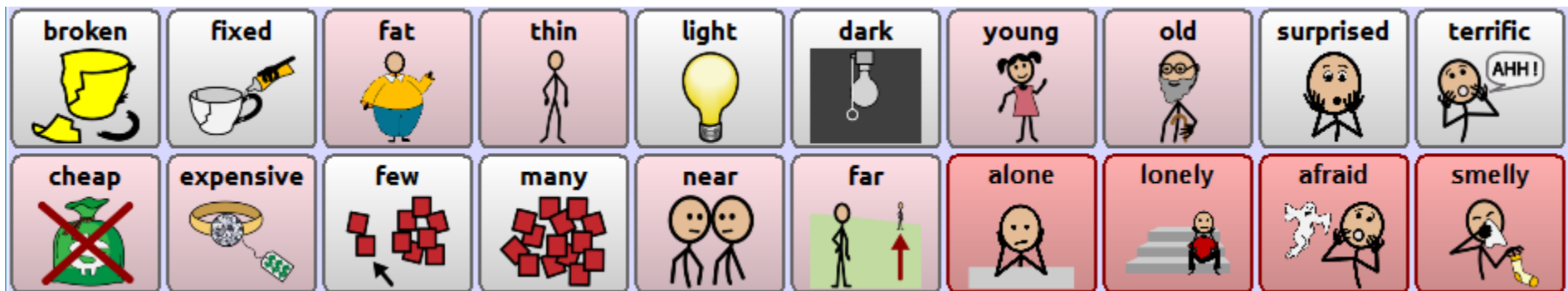













30



31




















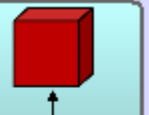

<b>red</b> 	<b>orange</b> 	<b>yellow</b> 	<b>green</b> 	<b>blue</b> 	<b>purple</b> 	<b>pink</b> 	<b>black</b> 	<b>white</b> 	<b>brown</b> 
<b>grey</b> 	<b>gold</b> 	<b>silver</b> 	<b>tan</b> 	<b>black &amp; white</b> 	<b>brown &amp; white</b> 				

**good**  


**more**  


**COLORS**  


<b>there</b> 	<b>away</b> 	<b>this</b> 	<b>again</b> 	<b>around</b> 	<b>front</b> 	<b>back</b> 	<b>behind</b> 	 <b>top</b>	 <b>over</b>
<b>not</b>	<b>don't</b> 	<b>between</b> 	<b>middle</b> 	<b>through</b> 	<b>or</b> 	<b>left</b> 	<b>right</b> 	<b>bottom</b> 	 <b>under</b>

ABC  
123

clear

.

EXTRA  
And It The  
WORDS  
Up