

WordPower 25 Touch & Scan SS practice

Reasons to Communicate!

- (R) Request
- (D) Direct
- (C) Comment
- (G) Greet
- (P) Protest
- (RP) Relay past events
- (A) Ask




*NOTE- WP 25 Touch & Scan was designed for someone possibly needing larger buttons, use of phrases and easier access. This file is also designed for scanning. Some of the practice sentences here would not be typical targets, but are used to demonstrate where vocabulary might be found.

(R) I want a drink.



(D) Go over there.

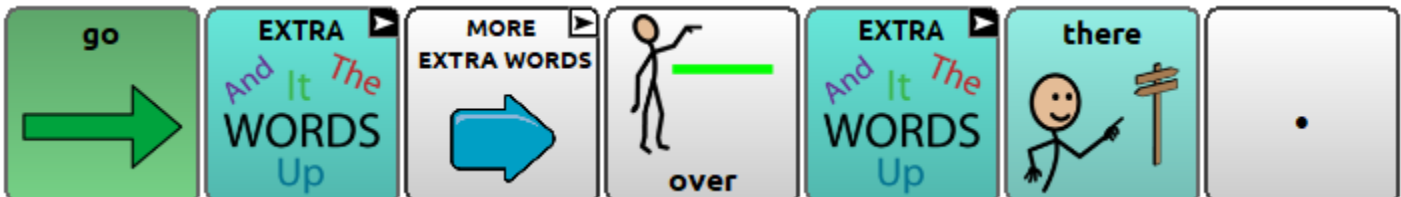


(C) I like it.




(G) Hello, How are you?



(P) Don't you look at me.

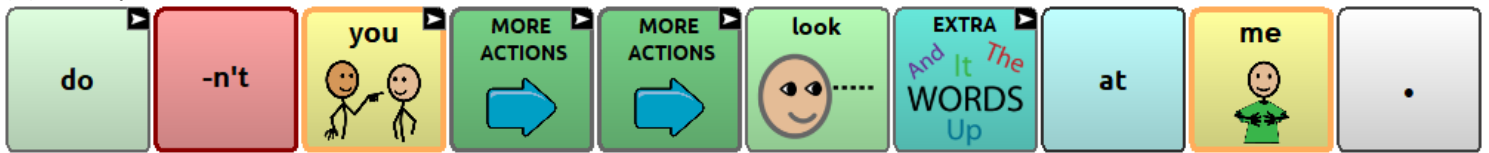

(RP) I played Legos.

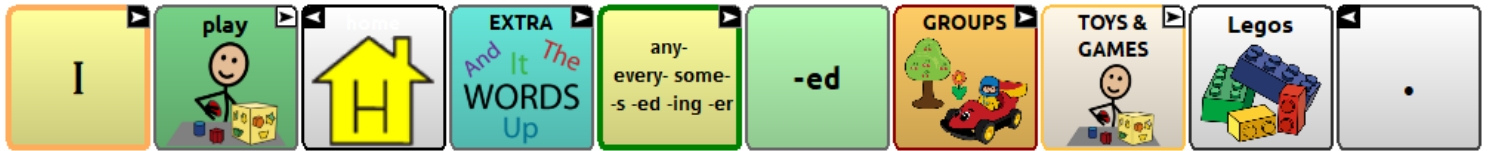

(A) Do you like to watch TV?




On your own:

- (R) I want to eat pretzels.
- (D) Turn it up.
- (C) I feel happy.
- (G) What's up?
- (P) Stop it.
- (RP) I went (to the) park.
- (A) Can I do it?

Exploring other features (some sequences provided, some to find on your own-just like our clients!):

Actions A-Z

Did you order?

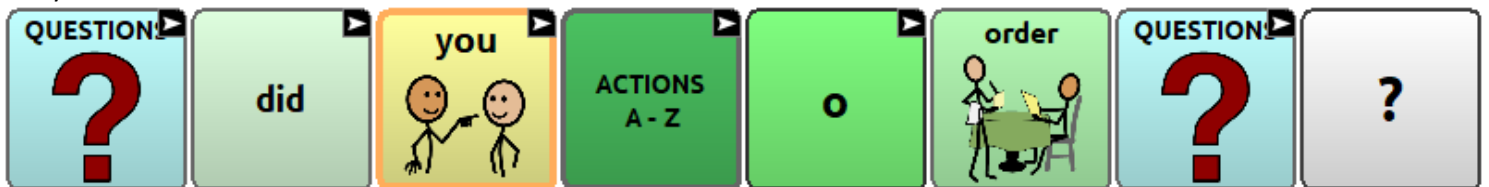

I am going to bake a chocolate cake.

Describe A-Z


She is amazing.



He is messy.

Logical next words

That is the favorite color.

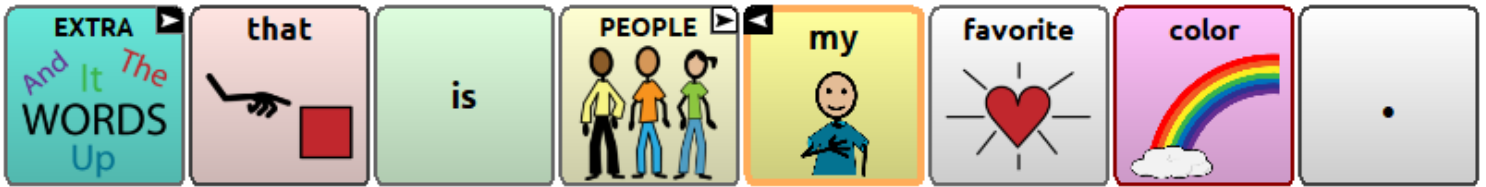

My turn.

Listen to music. (not available as next logical word, but does appear on next logical page)

Categories

Time to go.



Happy Birthday!



Something hurts. It's my head.



What will we do?

That is funny!

Put it down.