# WordPower120 Français

by Nancy Inman

Word  
power

Copyright © 2021  
Nancy L. Inman, M.A.T., CCC-SLP  
Speech-Language Pathologist


\*\*\***Dynamic Grammar Labels** should be **ON**  
(to turn ON, go to **Settings** > **Buttons** > **Dynamic Grammar Labels**)

## *Let's get to know the WordPower vocabulary design*

- 1) **Spelling** These vocabularies include a QWERTY keyboard on the main page, allowing one to move easily between core vocabulary words, and spelling with word prediction. (There is also an AZERTY keyboard version available.)

## 2) **Category Links**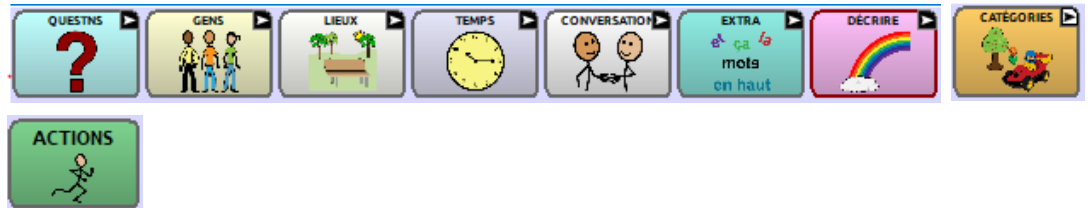

- a - **QUESTIONS** Question words. Words often lead to word completion options or logical next words. A smart “?” or smart “!” will add punctuation and speak a sentence.
- b – **GENS** A page set up to be customized with names of friends and family. Link to **GENS À L'ÉCOLE**, and **TRAVAIL**.
- c – **LIEUX** Common places programmed by subcategories of **buildings, home, outside, and school**. Links to **GÉOGRAPHIE, MAGASINS** and **RESTAURANTS**.
- d – **TEMPS** Time related words. “Aujourd'hui, c'est...” etc. opens a set up pages to tell the date. Link to  page to tell the time.
- e – **CONVERSATION** Greetings, interjections, social language, **PERSONNEL, QUESTIONS, TEXTING**.
- f – **DÉCRIRE** Describing words, colors, alphabetized and coded by opposites. Includes access to A to Z list of adjectives.
- g – **EXTRA MOTS** Extra high frequency words.
- h – **ACTIONS** Verbs programmed in alphabetical order. **A-Z** listing provides pages of actions categorized by initial letter.
- i – **CATÉGORIES** A page of semantic categories, etc.

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DEVICE TOOLS & FUN – accessed from the second page of **GROUPS**

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## \*\*LITERACY SUPPORTS


- Save written text from Message Window using “SAUVEGARDER MESSAGE” feature (Maj page) and/or the “HISTOIRES & PHRASES” feature (CATÉGORIES > HISTOIRES & PHRASES pages)
- **CONCEPTS DE L'HISTOIRE** and **TRAITS DE CARACTÈRE** available on the LECTURE page. These concepts are part of the Common Core curriculum requirements used by many school districts.
- Easy access to texting & sharing text - CONVERSATION page, Speech Display Bar or

### 3) Describing Words –take a look


The first page of DÉCRIRE words consists of “feeling words” and adjectives that are paired by opposites. The DÉCRIRE A-Z key provides access to less frequently used adjectives. Notice, the masculine/feminine toggle keys.

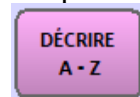### 4) Verbs –take a look

In addition to core verbs, verbs are accessed on the ACTIONS pages, and less frequently used verbs can be accessed by selecting the ACTIONS A-Z key on the ACTIONS page.

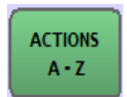## VERB FORMS

(try out these key elements with the verb “**prendre**” found on the main page)

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## Imperative verbs

(present tense 2nd singular form provided)

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Pronoun +

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+ ACTION word

- Je ( <b>imparfait</b> ) jouais avec toi.	- <i>I was playing with you.</i>
- Je ( <b>conditionnel</b> ) jouerais avec toi.	- <i>I would play with you.</i>
- Je ( <b>futur</b> ) jouerai avec toi.	- <i>I will play with you.</i>
- Je le ( <b>imparfait</b> ) cherchais.	- <i>I was looking for him.</i>
- Je le ( <b>conditionnel</b> ) chercherais.	- <i>I would look for him.</i>
- Je le ( <b>futur</b> ) chercherai.	- <i>I will look for him.</i>

**Verbs on Main Page** - double hit = infinitive form; When you select a verb on main page:

- a) top row consists of present tense forms
- b) left column provides inverted question forms
- c) “ne” included to provide negative form
- d) for some verbs, second person + moi included  
(i.e. dis-moi, dites-moi, etc.)

Verbs change automatically according by context:

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Direct links to associated category pages provided after verb selected

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- 5) **reset** \*\*\*Notice that whenever you need to reset the main page back to square one, you can select “**reset**,” which appears where the “**espace**” key normally appears

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### 9) Customized word predictor for high frequency words

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**11) Core vocabulary has been incorporated into each of the category pages. Try out some of these sentences, starting within a category. Notice that the same motor pattern is used whether speaking the sentence from the main page or from within the category.**

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**SHAPES & COLORS**

*I want a red.  
I love orange..  
I don't have blue.  
I want the green star.  
What color do you want?*

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**PETS**

*I want to feed my dog.  
Can I pet the cat?  
She is very cute.  
What is she doing?  
She is eating.*

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**ART**

*I want to color a picture please.  
It is very good.  
Do you like to paint?  
She is painting him a picture.*

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**BODY**

*My ear is hurting.  
My stomach. It is not hurting.  
I hurt my finger.  
I'm hurting.  
I'm not hurting.*

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**DRINKS**

*I would like to order a small soda please.  
  
Do you want to drink water?  
I like to drink hot tea with cream.  
I drank more chocolate milk.  
She can't drink iced tea.*

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**FOOD - SNACKS**

*I want to eat pizza with cheese and pepperoni please*

*Do you like chips with salsa?  
What do you want to eat for dinner?  
I want to eat lunch at Taco Bell.*

**READING**

*I want to read this book again please.  
I like to read stories about \_\_\_\_.  
Do you want to read to me?  
What are you reading?  
I'm going to read a book to you.*

**TV & MOVIES**

*I really love to watch cartoons.  
I would love to watch cartoons.  
Do you want to watch Netflix?  
Change the channel please.  
Do you want to watch a movie with me?  
They want to watch a Disney film.*