WordPower60 Español Básico – tutorial and reference guide

Nancy Inman & Pamela Corley

***On the main page, note the following:

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1. Access to Quick Basic Communication with Picture Support:

Example a) Notice that when selecting the pronouns, the verb format automatically agrees.

As single hits on the main page, however, you'll see that several of the core verbs appear in the imperative form (Imp), or the first-person form (FP). These high frequency verb forms were selected because of their usefulness as *sentence starters*.

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Example c) me gusta *“it pleases me”* (similar to *“I like”* and *“I would like”* in English)
me gustaría *“it would please me”* (notice link to **COSAS FAVORITAS** *“favorite things”*)

Me gusta (COSAS FAVORITAS + caballo) montar a caballo. <i>“I like to go horseback riding.”</i>	Me gusta escuchar música. <i>“I like to listen to music.”</i>
Me gusta comer en Chick-fil-A y Taco Bell. <i>“I like to eat at Chick-fil-A and Taco Bell.”</i>	Me gusta jugar contigo. <i>“I like to play with you.”</i>
Me gustan mis (GENTE) amigos. <i>“I like my friends.”</i>	Me gustaría leer un libro. <i>“I would like to read a book.”</i>

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Tengo mucha hambre <i>“I am really hungry.” (literally “I have hunger.”)</i>	Tengo sed. <i>“I am thirsty.” (literally “I have thirst.”)</i>
Tengo sueño. <i>“I am sleepy.”</i>	Tengo frío. <i>“I am cold.” (literally “I have cold.”)</i>
Tengo un problema. <i>“I have a problem.”</i>	

Example e) me siento *“I feel”* (literally *“I feel myself”*)
 (link to **DESCRIPCIÓN**). Notice “m” for masculine and “f” for feminine

Me siento triste. <i>“I feel sad.”</i>	Me siento cansado (masc). Me siento cansada (fem). <i>“I feel tired.”</i>
---	--

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2. Direct Link from Verbs to Related Categories:

Example a) comer "to eat"

Quiero comer pizza con pepperoni y cebolla. <i>"I want to eat pizza with pepperoni and onion."</i>	Quiero comer contigo. <i>"I want to eat with you."</i>
Me gusta comer huevos con tocino. <i>"I like to eat eggs with bacon."</i>	Nosotros comemos palomitas de maíz. <i>"We eat popcorn."</i>

Example b) beber "to drink"

"Quiero beber Coca-Cola y un café con crema." <i>"I want to drink Coca-Cola and coffee with cream."</i>	Quiero beber un poco de limonada. <i>"I want to drink a little lemonade."</i>
Me gusta beber <u>refresco</u> de naranja. <i>"I like to drink orange soda."</i>	Quiero beber <u>jugó natural</u> de sandía. <i>"I want to drink watermelon juice."</i>
***Since the word "tomar" is so commonly used for drinking, if you say "quiero tomar," you'll see that links are provided to the BEBIDAS page.	

Example c) jugar "to play" (on JUGUETES page, you'll find links to JUEGOS "games," and DEPORTES "sports")

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3. Easy Access to Noun Categories in GRUPOS (word categories):

Mi bicicleta es azul. "My bicycle is blue."	hint: Find "bicicleta" in VEHÍCULOS or JUGUETES, and from that page COLORES for "azul."
Quiero ir a Hawaii. "I would like to go to Hawaii."	hint: Choose "quiero" on main page. Find "Hawaii" in GEOGRAFÍA (LUGARES →GEOGRAFIA) (NOTE: "a" and "al" appear automatically following "ir")
Tengo un auto y una camioneta. "I have a car and a pickup truck."	hint: Choose "tengo" on main page. Find "auto" and "camioneta" in VEHÍCULOS.
Me siento feliz porque puedo ir al concierto. "I feel happy because I can go to the concert."	hint: Choose MÚSICA to find "concierto."
Es mi canción favorita. "That is my favorite song."	hint: First, select the MÚSICA category. You will be able to speak this entire sentence from that category.

4. Verb forms automatically agree with Pronouns selected:

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Example c) Second page of ACCIONES “bailar” (“to dance”)

(hint: navigate using the arrow in bottom right of screen)

Yo bailo. “I dance.”
Tú bailas. “You dance.”
Él baila. “He dances.”
Ellos bailan. “They dance.”
Nosotros bailamos. “We dance.”

Example d) ACCIONES A-Z (hint: select the pronoun and then the “ACCIONES A-Z” link)

Yo <u>conduzco</u> . “I drive.”	
Ella <u>pelea</u> conmigo. “She fights with me.”	(hint: look for “con” + “-migo” = “conmigo”)
Ellos <u>disfrutan</u> de la playa. “They enjoy the beach.”	(hint: look for “playa” in LUGARES)
Nosotros <u>montamos a caballo</u> en <u>la mañana</u> . “We ride horses in the morning.”	(hint: look for “a caballo” following “montamos;” look for “la mañana” in TIEMPO)

Notice there is an additional option with ACCIONES A-Z. In addition to choosing the pronoun on the main page and navigating to ACCIONES A-Z, once inside ACCIONES A-Z page you may select the pronoun on the left in yellow, and the verb forms change accordingly.

5. Verb forms included in WordPower60 Español Básico:

Present tense

Preterite *(Past tense verbs)*

Future *(express the future verb form by saying, "Yo voy a ___." (I am going to ___.)*

Infinitive *(to + verb)*

Present participle *(-ing)*

Imperativo *(Imperative verbs - find this link on the ACCIONES page)*

6. Verb forms appear in correct format automatically by semantic context:

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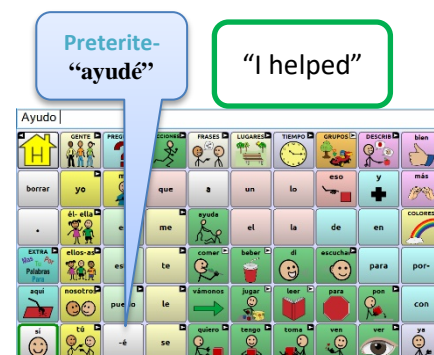
Example c) Yo voy a ("I am going") + infinitives

Yo voy a leer. "I'm going to read."	Yo voy a jugar con mi iPad. "I'm going to play with my iPad."	Yo voy a comer. "I'm going to eat."
Yo voy a comprarlo. (comprar + lo). "I'm going to buy it."		

7. To access the preterite verb tense (past tense), select the pronoun and present tense of the verb. The preterite form appears just to the right of the “tú” key.

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8. Core vocabulary included within categories:

The following sentences demonstrate the feature of core vocabulary included within categories.

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9. Masculine/feminine nouns agree with Adjectives:

In Spanish, adjectives must match the noun in terms of number and gender. This is done automatically when choosing a color to modify a noun in categories such as ANIMALES (*animals*), VEHÍCULOS (*vehicles*), and ROPA (*clothes*). ****Notice that the gender and number of the color changes automatically to agree with the nouns, and that the noun is positioned before the adjective.**

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10. Plural fun!

In Spanish as in English, plural markers can be final –s or final –es depending upon the final letter of the noun. *****After selecting a noun, look for "-s" in the left column.**

- Try pluralizing these words: "pez" (*fish*), "libro" (*book*), "dólar" (*dollar*)

With two or three word noun concepts, the plural marker is typically on the first word. Choose the **TRABAJOS** category for the following examples:

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12. PREGUNTAS (Questions):

In Spanish, the proper punctuation for questions is an upside down question mark (¿) at the beginning of the sentence and a regular question mark (?) at the end. When starting a question from the PREGUNTAS page, the "¿" appears automatically with WordPower60 Español Básico.

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13. Using the Imperative form of the Verb:

**Some imperatives appear on the main page, as well as on the first ACCIONES page. Others are found by selecting “ACCIONES+IMPERATIVO” or “ACCIONES+INFINITIVO+IMPERATIVO”

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14. Other features in WordPower:

1. Select the GRUPOS key to find the CHISTES (Jokes). There are jokes and riddles. Try them out.
2. Select the GRUPOS key to find the NOTICIAS (News).
3. Select the GRUPOS key to find FOTOS. This gives access to a photo album that can be used to share pictures and experiences.

15. PUTTING IT ALL TOGETHER: try these sentences to practice your skills:

Me gusta comer en la taquería. “I like to eat at the taqueria.”	Me gusta comer en casa. “I like to eat at home.”	Me gusta comer en McDonald’s. “I like to eat at McDonald’s.”
Le gusta comer conmigo. “He/she likes to eat with me.”	¿Qué te gusta comer? “What do you like to eat?”	¿Quieres comer conmigo? “Do you want to eat with me?”
Tengo un perro y dos gatos en mi casa. (Hint: find “perros” and gata” in ANIMALES; “casa” in LUGARES) “I have one dog and two cats at my house.”		
Yo fui con ella. “I went with her.”	(Hint: “fui” follows “yo” on main page; select “él-ella” to find “ella”)	
Ya está. “That’s it.”	Ya vámonos “Let’s go.”	
Tengo algo para ti. “I have something for you.”	(Hint: find “algo” on the EXTRA PALABRAS page)	
No me siento bien. “I don’t feel well.”		

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