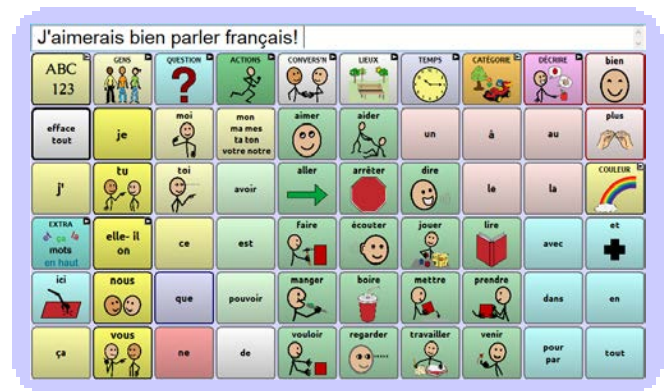
WordPower60 Français

by Nancy Inman

*Word
power*

Copyright © 2021
Nancy L. Inman, M.A.T., CCC-SLP
Speech-Language Pathologist


*****Dynamic Grammar Labels** should be ON
(to turn ON, go to **Settings > Buttons > Dynamic Grammar Labels**)

Let's get to know the WordPower vocabulary design

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***Note, when you select **efface tout** or **.** the **j'** becomes available.

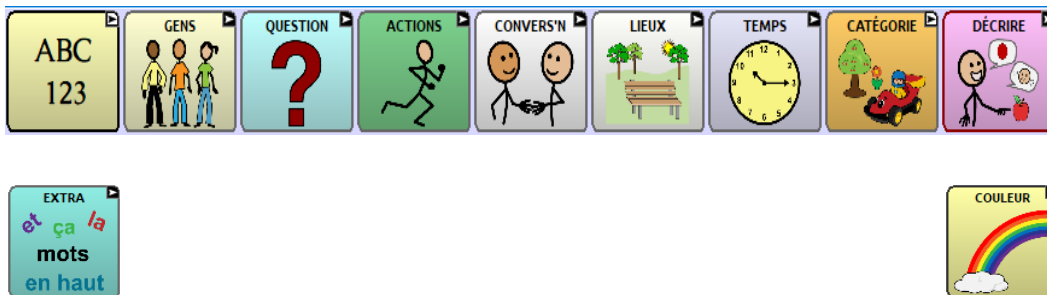1) Category Links

- a – SPELLING Spelling and numbers available from this page. There are links to simplified spelling and numbers pages.
- b – GENS A page set up to be customized with names of friends and family.
Link to **GENS À L'ÉCOLE**, and **TRAVAIL**.
- c - QUESTIONS Question words. Words often lead to word completion options or logical next words.
A smart “?” or smart “!” will add punctuation and speak a sentence.
- d – ACTIONS Verbs programmed in alphabetical order.
A-Z listing provides pages of actions categorized by initial letter.
- e – CONVERSATION Greetings, interjections, social language, **PERSONNEL & QUESTIONS, TEXTING**

f – LIEUX Common places programmed by subcategories of **buildings, home, outside,** and **school.**
Links to **MAGASINS, RESTAURANTS,** and **GÉOGRAPHIE.**

g – TEMPS Time related words. “Aujourd’hui, c’est...” etc. opens a set up pages to tell the date.
Link to  page to tell the time.

h – DÉCRIRE Describing words - alphabetized and coded by opposites.
Includes access to A to Z list of adjectives.

i – EXTRA mots Extra high frequency words.

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**LITERACY SUPPORTS

- Save written text from Message Window using “**SAUVEGARDER MESSAGE**” feature on the Page / or the “**HISTOIRES & PHRASES**” feature (**CATÉGORIES > HISTOIRES & PHRASES** pages)
- Book pages and book templates for young learners - (**CATÉGORIES > LECTURE > PREMIERS LIVRES**)
- **CONCEPTS DE L'HISTOIRE** and **TRAITS DE CARACTÈRE** available on the **LECTURE** page. These concepts are part of the Common Core curriculum requirements used by many school districts.

- Easy access to texting & sharing text - **CONVERSATION** page, Speech Display Bar or found on the top row of the **CATÉGORIES** page.

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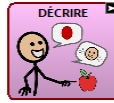
- Beginning Keyboard and Numbers pages   found on the  page.


LOWTECH BOARDS

- Select  and then  to find manual communication boards that can be printed.

2) Describing Words –take a look

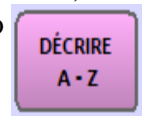
The first page of DÉCRIRE words consists of “feeling words” on the right side of the screen, and adjectives that are paired by opposites. The DÉCRIRE A-Z key provides access to less frequently used adjectives. Notice, the masculine/feminine toggle keys.

3) Verbs –take a look

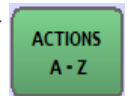In addition to core verbs, verbs are accessed on the ACTIONS pages, and less frequently used verbs can be accessed by selecting the ACTIONS A-Z key on the ACTIONS page.

VERB FORMS

(try out these key elements with the verb “prendre” found on the main page)

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Imperative verbs

(present tense 2nd singular form provided)

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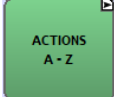
***Dynamic Grammar Labels must be ON

Pronoun +  +    + ACTION word

- Je + A - Z + (imparfait)	jouais avec toi	- <i>I was playing with you.</i>
- Je + A - Z + (conditionnel)	jouerais avec toi	- <i>I would play with you.</i>
- Je + A - Z + (futur)	jouerai avec toi	- <i>I will play with you.</i>
- Je + A - Z + (imparfait)	cherchais	- <i>I was looking for him.</i>
- Je + A - Z + (conditionnel)	chercherais	- <i>I would look for him.</i>
- Je + A - Z + (futur)	chercherai	- <i>I will look for him.</i>

Verbs on Main Page - double hit = infinitive form; When you select a verb on main page:

- a) top row consists of present tense forms
- b) left column provides inverted question forms
- c) “ne” included to provide negative form
- d) for some verbs, second person + moi included
 - i.e. dis-moi, dites-moi
 - écoute-moi, écoutez-moi
 - prends-moi, prenez moi
 - etc.

Verbs change automatically according by context:

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Direct links to associated category pages provided after verb selected

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When you select a verb on the main page, the left column provides inverted question forms.

Helpful hint – all verbs on the two main **ACTIONS pages that end in **-er** have a special option which allows the “**-ez-vous**” question form to be generated:

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elle- il
on

> lead to il(s), elle(s), and common phrases beginning with "il"

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J'ai déjà vu ça.
I have already seen that.

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Je ne pense pas que vous allez **ACTIONS** trouver ça.
I don't think you're going to find that.

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8) Core vocabulary has been incorporated into each of the category pages. Try out some of these sentences, starting within a category. Notice that the same motor pattern is used whether speaking the sentence from the main page or from within the category.

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SHAPES & COLORS

I want a red.
My favorite color is purple.
My favorite shape is a star.
I love orange..
I don't have blue.
I want the green star.
What color do you want?

PETS

I want to feed my dog.
Can I pet the cat?
She is very cute.
What is she doing?
She is eating.

ART

I want to color a picture please.
It is very good.
Do you like to paint?
She is painting him a picture.

BODY

My ear is hurting.
My stomach. It is not hurting.
I hurt my finger.
I'm hurting.
I'm not hurting.
I hurt myself.

DRINKS

I would like to order a small soda please.
Do you want to drink water?
I like to drink hot tea with cream.
I drank more chocolate milk.
She can't drink iced tea.

FOOD - SNACKS

I want to eat pizza with cheese and pepperoni please.
Do you like chips with salsa?
What do you want to eat for lunch?
I want to eat dinner at Taco Bell.

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READING

I want to read this book again please.

I like to read stories about ____.

Do you want to read to me?

What are you reading?

I'm going to read a book to you.

TV & MOVIES

I love to watch cartoons.

I would love to watch cartoons.

Do you want to watch Netflix?

Change the channel please.

Do you want to watch a movie with me?

They want to watch a Disney film.