wordpower®

WordPower25 Español

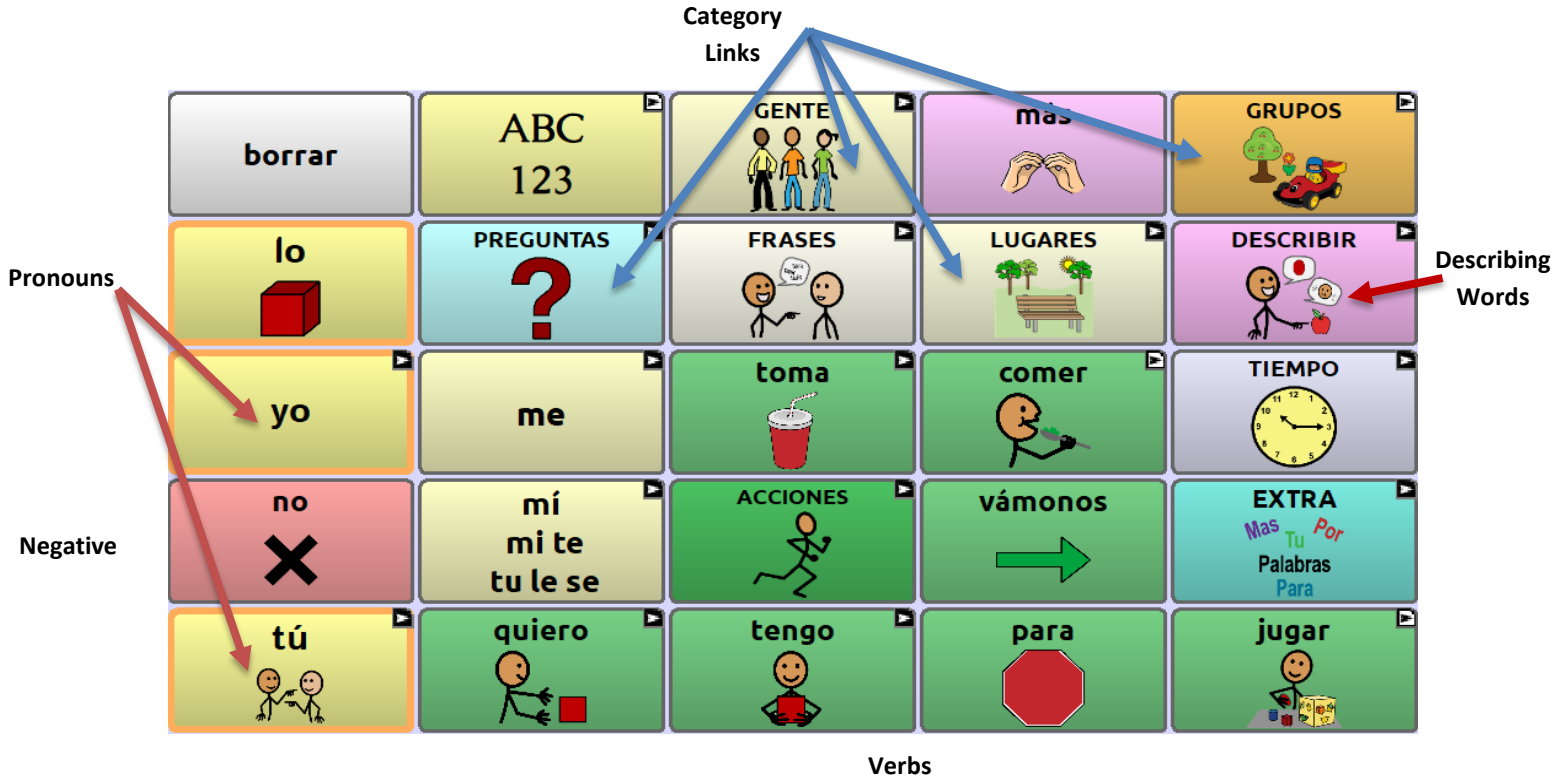
Tutorial and Reference Guide

Nancy Inman & Pamela Corley

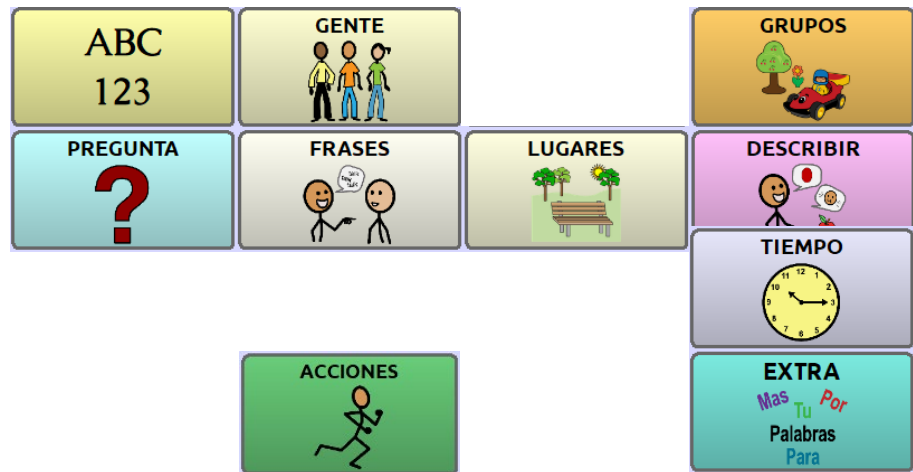
Contents

1. Category Links	4
2. Describing Words: Take a look.....	6
3. Verbs: Take a look	6
4. Access to Quick Basic Communication with Picture Support	7
5. Direct Link from Verbs to Related Categories.....	10
6. Easy Access to Noun Categories in GRUPOS (word categories)	11
7. Verb Forms Appear in Correct Format Automatically by Semantic Context	12
8. Future.....	12
9. Core Vocabulary Included within Categories.....	13
10. Masculine/Feminine Nouns Agree with Adjectives	14
11. Logical Next Words	14
12. PREGUNTAS (Questions).....	15
13. Other features in WordPower	15
14. Putting It All Together: Try these sentences to practice your skills.....	16

Let's get to know the WordPower25 Español vocabulary design

1. Category Links

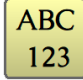
- a –  25-location spelling and numbers available from this page.
- b – **GENTE** A page set up to be customized with names of friends and family.
Link to **PERSONAL ESCUELA**, and **TRABAJOS**.
- c – **PREGUNTAS** A page set up to be customized with names of friends and family.
Link to **PERSONAL ESCUELA**, and **TRABAJOS**.
- c – **PREGUNTAS** Question words. Words often lead to word completion options or logical next words.
A smart “?” or smart “!” will add punctuation and speak a sentence.
- d – **FRASES** Greetings, interjections, social language, **PERSONAL Y PREGUNTAS**, **TEXTING**
- e – **LUGARES** Common places programmed. Links to **TIENDAS**, **RESTAURANTES & GEOGRAFIA**
- f – **DESCIBIR** Describing words - alphabetized and coded by opposites. Feminine and masculine.
Includes access to A to Z list of adjectives.
- g – **TIEMPO** Time related words.  Link to a page to tell time.
- h – **ACCIONES** Verbs programmed in alphabetical order. **A-Z** listing provides pages of actions categorized by initial letter.
- i – **EXTRA** words Extra high frequency words.
- j – **GRUPOS** Two pages of semantic categories, etc.

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**LITERACY SUPPORTS

- Book pages and book templates for young learners
GRUPOS > LECTURA > LIBROS INICIALES
- Phonetics page - **FONÉTICA ESPAÑOLA**
- Easy access to texting & sharing text - **FRASES** page (**TEXTO**), Speech Display Bar, and found on the **EMOJIS** page
- Save written text from Message Window using “**GUARDAR FRASE**,” accessed on the  page, or the “**HISTORIAS Y FRASES**” page
-  “Escribir on Apps” allows one to open iOS or Android apps from your vocabulary, and copy text from the Message Window into the app

LOWTECH BOARDS

- Select  and then  to find manual communication boards that can be printed.

2. Describing Words: Take a look

The first page of DESCRIBIR words consists of “feeling words.” Adjectives on subsequent pages are paired by opposites. The ADJECTIVO A-Z key provides access to less frequently used adjectives. Notice the masculine/feminine toggle keys.

3. Verbs: Take a look

In addition to core verbs, verbs are accessed through the ACCIONES pages, and less frequently used verbs can be accessed by selecting the ACTIONS A-Z key on the ACCIONES page.

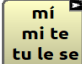
VERB FORMS

Present tense	- yo juego, tú juegas, ella juega, ellos juegan, nosotros jugamos, vosotros jugáis
Present participle	- appears automatically in context ("yo estoy comiendo," "estoy viniendo," "estoy jugando, etc.")
Infinitive	- ACCIONES > INFINITIVO - appear automatically in context (e.g. "quiero __," "me gusta __," "estoy listo para __," "necesito __," "amo __")
Future	- express the concept of future by saying, "Yo voy a __." (I am going to __.)
Subjunctive (tú)	- "Quiero que (tú) __," "Necesito que (tú) __," "Espero que (tú) __" "Quiero que <u>tomes</u> una foto." (I want you to take a picture.) "(Yo) necesito que <u>vengas</u> aquí." (I need you to come here.) "(Yo) espero que <u>termines</u> ." (I hope you finish.)
Imperativo	(Imperative verbs – ACCIONES > IMPERATIVO)

4. Access to Quick Basic Communication with Picture Support

Example a) Notice that when selecting the pronouns, the verb format automatically agrees.

As single hits on the main page, however, you'll see that several of the core verbs appear in the imperative form (Imp), or the first-person form (FP). These high frequency verb forms were selected because of their usefulness as *sentence starters*.

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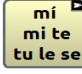
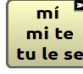
Example c) me gusta *“it pleases me”* (similar to *“I like”* and *“I would like”* in English)
me gustaría *“it would please me”* (notice link to **COSAS FAVORITAS** “favorite things”)

Me gusta (COSAS FAVORITAS + caballo) montar a caballo. <i>“I like to go horseback riding.”</i>	Me gusta ➡ escuchar música. <i>“I like to listen to music.”</i>
Me gusta comer en Chick-fil-A y Taco Bell. <i>“I like to eat at Chick-fil-A and Taco Bell.”</i>	Me gusta jugar contigo. <i>“I like to play with you.”</i>
Me gustan mis (GENTE) amigos. <i>“I like my friends.”</i>	Me gustaría leer un libro. <i>“I would like to read a book.”</i>
 A ella le gusta ver una película. <i>“She likes to watch a movie.”</i>	 A ellos les gustaría eso. <i>“They would like that.”</i>

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Tengo mucha hambre <i>“I am really hungry.” (literally “I have hunger.”)</i>	Tengo sed. <i>“I am thirsty.” (literally “I have thirst.”)</i>
Tengo sueño. <i>“I am sleepy.”</i>	Tengo frío. <i>“I am cold.” (literally “I have cold.”)</i>
Tengo un problema. <i>“I have a problem.”</i>	

Example e) me siento *“I feel”* (literally *“I feel myself”*)
 (link to **DESCRIBIR**). Notice “m” for masculine and “f” for feminine

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Example g) FRASES (quick access to often-used phrases)

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5. Direct Link from Verbs to Related Categories

Example a) comer "to eat"

Quiero comer ➡ pizza con pepperoni y cebolla. "I want to eat pizza with pepperoni and onion."	Quiero comer contigo. "I want to eat with you."
Me gusta comer GRUPOS DE COMIDA > DESAYUNO huevos y tocino. "I like to eat eggs and bacon."	(GENTE) Nosotros comemos galletas saladas. "We eat crackers."

Example b) tomar "to take" "to drink"

***Since the word "tomar" is so commonly used for drinking, if you say "quiero tomar," you'll see that links are provided to the BEBIDAS page.	No me gusta (BEBIDAS) tomar soda. "I don't like to drink soda."
Quiero tomar (BEBIDAS) jugo y leche chocolate. "I want to drink juice and chocolate milk."	Quiero tomar (BEBIDAS) limonada por favor. "I want to drink lemonade please."
Me gusta tomar (BEBIDAS) ➡ <u>refresco</u> (SABORES) de naranja. "I like to drink orange soda."	Quiero tomar (BEBIDAS) ➡ <u>jugo natural</u> (SABORES) de sandía. "I want to drink watermelon juice."
Quiero tomar una foto. "I want to take a photo."	

Example c) jugar "to play" (on **JUGUETES** page, you'll find links to **JUEGOS** "games," and **DEPORTES** "sports")

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6. Easy Access to Noun Categories in GRUPOS (word categories)

(VEHÍCULOS) La bicicleta es azul. “The bicycle is blue.”	hint: Find “bicicleta” in VEHÍCULOS or JUGUETES , and from that page COLORES for “azul.”
Quiero ir a Hawaii.	hint: Find GEOGRAFÍA on the second page or LUGARES (NOTE: “a” and “al” appear automatically following “ir”)
(VEHÍCULOS) Tengo un auto y una camioneta.	hint: Select “yo” in VEHÍCULOS and “tengo” appears.
Me siento feliz porque (yo) puedo ir al parque. “I feel happy because I can go to the park.”	hint: “porque” appears automatically after speaking a feeling
Mi color favorito es el azul. “My favorite color is blue.”	hint: First, select DESCRIBIR and then COLORES . You will be able to speak this entire sentence from that category.

7. Verb Forms Appear in Correct Format Automatically by Semantic Context

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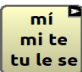
9. Core Vocabulary Included within Categories

The following sentences demonstrate the feature of core vocabulary included within categories.

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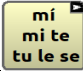
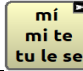
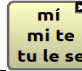
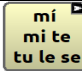
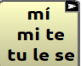
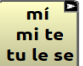
10. Masculine/Feminine Nouns Agree with Adjectives

In Spanish, adjectives must match the noun in terms of number and gender. This is done automatically when choosing a color to modify a noun in categories such as **ANIMALES** (animals), **VEHÍCULOS** (vehicles), and **ROPA** (clothes).

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12.PREGUNTAS (Questions)

In Spanish, the proper punctuation for questions is an upside down question mark (¿) at the beginning of the sentence and a regular question mark (?) at the end. When starting a question from the PREGUNTAS page, the “¿” appears automatically.

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13.Other features in WordPower

1. Select the **GRUPOS** key to find the **CHISTES** (Jokes). There are jokes and riddles. Try them out.
2. Select the **GRUPOS** key to find the **NOTICIAS** (News).
3. Select the **GRUPOS** key to find **FOTOS**. This gives access to a photo album that can be used to share pictures and experiences.

14. Putting It All Together: Try these sentences to practice your skills

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