

Available Button Actions

Button Name	Description
No Action	Performs no action
Add Message to Display	Inserts text from the button's message box into the display, but the words are not spoken
Add Time/Date to Display	Displays the current date and time in the Speech Display Bar (SDB)
Apply Modifier	A modifier key is a special key on a keyboard that modifies the normal action of another key. (Caps Lock, Ctrl, Functions, Num Lock, Shift)
Backspace key	Deletes the last character in the display
Battery Status	Displays the current battery status in the SDB
Cancel Visit	Clears all remaining visits
Clear All Modifiers	Releases any Shift, Caps, Ctrl or Num Lock modifiers.
Clear Display	Clears the text in the SDB
Clear Last Word	Deletes the last word in the SDB
Data Logging On/Off	Device usage logs for language analysis
Find Word	Finds the path to a word you are looking for. When you tap a button and then tap Find Word, the path to the word is displayed in the SDB.
Grammar Action	Changes the existing word to the chosen grammatical form, add -ed, add -en, add -er, add -est, add -ing, Add -s
Help Signal	Triggers a loud attention-getting sound
Jump to Page	Takes you to a new page and takes you back to the previous page after a single hit. Clears any unused Jumps
Load Display	Recalls text that had been saved and adds it to the display
Navigate	Takes you to a new page until you choose another button to move away from that page
Navigate back	Returns to the previously accessed page
Navigate to home	Takes you to the home page
Open Android Application	Provides a button from vocabulary page to open a specific Android application. (not functional if working with a Closed/Dedicated system)
Play audio	Plays audio files. These are recordings of someone speaking or sounds.
Play Library Audio	Plays an audio file such as music contained in the media library (Does not transfer with vocab file)

Button Name	Description
Play Library Video	Plays a video file contained in the media library (Does not transfer with vocab file)
Record Audio	Records audio files. These are recordings of someone speaking or sounds.
Repeat Last Spoken	Repeats the last spoken message
Save Display	Saves the text currently showing on the display for later use.
SDB – Copy Text	This action will copy all text within the Speech Display Bar and put a copy in the Clip Board
SDB – Paste Text	This will add text from the Clip Board to the end of the Speech Display Bar
SDB – Share Text	This action will copy all text from the Speech Display Bar to the chosen Social Network
Select Profile	Changes Settings to match the saved set of settings that was saved as a profile
Speak	Pronounces the SDB content
Speak Label Only	Speaks the label
Speak Last Sentence	Repeats the last spoken text
Speak Message Only	Speaks the message, but doesn't display the text
Speech Message	Types the button message in the SDB and pronounces it
Static Character Prediction (SCP)	Displays the most probable character in compliance with the current character displayed in the SDB, the assigned prediction order, and the static character list used by the dictionary compiler. You can adjust the prediction order.
Static Word Prediction (SWP)	Displays the most probable word in compliance with the current characters displayed in the SDB, the assigned selected prediction order, and the static word list used by the dictionary compiler (up to 60,000 words). The prediction order can be adjusted.
Stop	Stops the speech or audio file currently playing
Stop Library Audio	Stops the library audio
Store Text to Button	Stores text currently in the SDB to the next chosen button
Toggle Mute / Unmute	If the device is currently speaking a long message, selecting this button will not stop the speech but will stop the speech from being spoken aloud. Selecting the button again lets the text be heard.

Button Name	Description
Visit	Takes you to a new page and takes you back to the previous page after a single hit. Remembers unused Visits.
Visit Exception	If you have one button on a Visit page that you don't want to return to the previous page, use a Visit Exception action to leave it on the page for one additional hit.
Volume Down	Turns down the volume.
Volume Up	Turns up the volume.