



Devices, buttons, and features vary across devices, thus, make sure to note which buttons/options are on your device.

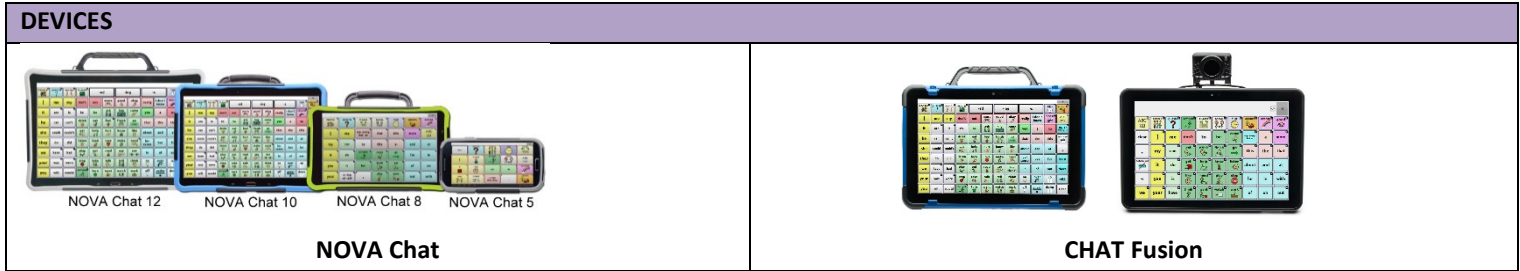

CHAT HARDWARE MENUS


The icons will vary based on which device you have.



Menu: allows you to determine which vocabulary file will be used, which augmentative options are enabled, how and when to speak messages, and how a wide variety of other options are configured.

CHAT MENU

When you select the Menu button, a pop-up menu is displayed at the bottom of the screen.



- Edit Mode:** turn editing on/off
- Settings:** modify settings to device
- Library:** listing of vocabulary files
- Help:** system information & registration
- Word Finder:** help finding a word (*must enable in Settings*)
- Lock:** lock menus (*must enable in Settings*)

Selecting a Vocabulary File

Tap the **Menu**


Then select **Library**

- All available vocabularies will be listed alphabetically.
- Locks indicate an original Salttilo file.
 - You CANNOT edit original files. You must duplicate it.



Duplicating a Vocabulary File

- Tap on the file you want to copy, select **Duplicate**.
- Type a name for the file. It is recommended to leave the vocabulary file name (e.g., MultiChat 15_Dave). **Save**.
- Tap on the newly created file. **Open**.

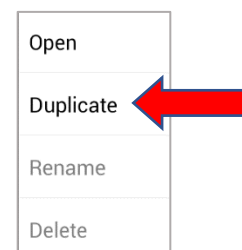

VOCABULARY CUSTOMIZATION

Note: The device will need to be in **Edit Mode** in order to make all changes described below.

Reminder about Edit Mode

- o **Menu ⇒ Edit Mode**
 - o **Edit Mode** will appear in red at the bottom of the screen, along with the name of the page (in brackets) you are currently editing.

CREATING AND EDITING PAGES IN A VOCABULARY

<p>Create a New Page</p>	<p>Menu ⇒ Pages ⇒ Menu ⇒ New Page</p> <ul style="list-style-type: none"> o Multiple things to customize: <ul style="list-style-type: none"> o Number of rows and columns o Background color or set a picture When finished customizing, select Save
<p>Linking to a New Page</p>	<p>Go to the page where the link will be created from. Identify and select the button that will link to a new page.</p> <ul style="list-style-type: none"> o Press and hold button, select Edit Button <ul style="list-style-type: none"> o Give the button a label and icon if needed. o Under the Actions tab, choose Add an Action o Scroll down to Navigate and select it <ul style="list-style-type: none"> o Select the New Page that was created. When finished press Save
<p>Add a Home Button to a New Page</p>	<p>If a Home Button already exists on another page, it can easily be created on a New Page.</p> <p>Can use this feature to add other buttons, as well, e.g. forward/back arrows</p> <ul style="list-style-type: none"> o Decide where the Home Button will be; then, press and hold the button. o Choose Add Button from Library o Select a page that has the desired button. o Scroll through the list to find desired button and select it.
<p>Create a New Page from Template</p>	<p>Menu ⇒ Pages ⇒ Menu ⇒ New from Template</p> <p>Select desired template (usually select the one that matches vocabulary file, e.g. 60-location, 42-location, etc.)</p> <ul style="list-style-type: none"> o Give the page a name When finished, select Save <p>Repeat steps to create buttons and link to new page where desired.</p>
<p>Importing Pages</p>	<p>Allows a person to import pages from a different vocabulary.</p> <p>Menu ⇒ Pages ⇒ Menu ⇒ Import</p> <ul style="list-style-type: none"> o Scroll through vocabularies and select the one where the pages will be imported from o Scroll through the pages and select all desired pages. o Once finished selecting pages, press Import. <p>It may take a moment to import all the pages. Once imported create new buttons and links as desired.</p>
<p>Editing Pages</p>	<p>Menu ⇒ Edit Page</p> <ul style="list-style-type: none"> o General Tab: change name of the page, color of the page, add an image o Layout Tab: adjust number of rows and columns


Notes about **Creating and Editing Pages**:

OTHER CUSTOMIZATIONS	
Abbreviations	<p>Helpful for a user who is adept at typing.</p> <p>Menu ⇒ Settings ⇒ Abbreviations</p> <p>To add a new abbreviation:</p> <ul style="list-style-type: none"> o Press Menu again o Then, New <ul style="list-style-type: none"> o Type in the desired abbreviation and what the expansion will be in the appropriate boxes. Select Save when finished.
Creating Gestures	<p>Gestures must be enabled in the Settings.</p> <p>Menu ⇒ Settings ⇒ Input ⇒ Gestures (check the box to enable)</p> <p>To create gestures:</p> <ul style="list-style-type: none"> o In the entire vocabulary: <ul style="list-style-type: none"> o Menu ⇒ Edit Vocabulary ⇒ Gestures New o On one specific page: <ul style="list-style-type: none"> o Menu ⇒ Edit Page ⇒ Gestures ⇒ New <ul style="list-style-type: none"> o Creating a New Gesture. Select which gesture will be used from the drop down menu. o Give the gesture a label and message. o Add an action from the pop up menu. This is what action will happen as a result of the gesture. o Select Save when finished.
Copying Pictures from Your Computer to the Device	<ul style="list-style-type: none"> o Attach device to computer. Allow for computer to recognize device. o Open Pictures folder. o Copy pictures from the computer (from desktop, picture folder, etc.) into the device's Pictures folder. o When the picture finish copying, unplug the device.
Creating a New Vocabulary	<p>Menu ⇒ Library ⇒ Menu ⇒ New</p> <ul style="list-style-type: none"> o Give the vocabulary a name and set the number of rows and columns. <ul style="list-style-type: none"> o When finished, select Save.
Notes about Other Customizations :	

CHAT EDITOR (formerly NOVA Chat Editor)

What to do with it

- Backup customized files
- Customize without taking the device away
- Create materials using Capture tool
- Import News-2-You & Unique files
- Transfer vocabulary to/from TouchChat
- Learn vocabulary

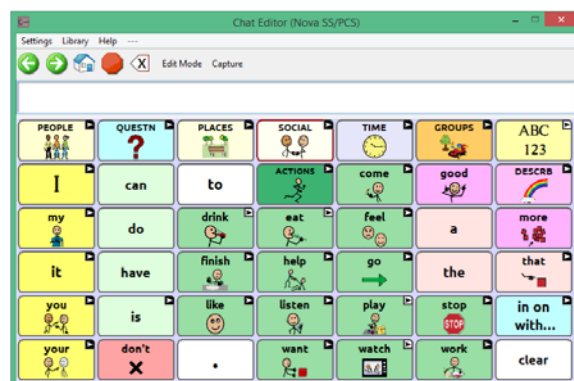

Free download available from www.salttillo.com

Available on all PCs (not compatible with Apple/iOs).

CHAT EDITOR	
TRANSFERRING FILES	
Connecting the Device to Chat Editor	<ul style="list-style-type: none"> o Start Chat Editor on the computer (must already be downloaded) o Make sure device is <u>not</u> in Edit Mode (Menu ⇒ Uncheck Edit Mode) <ul style="list-style-type: none"> o Attach device to the computer via included USB cable. Device will enter Transfer Mode. o "Device is Connected" will appear at the top of ChatEditor.
Transferring Files From a Device To Chat Editor	<ul style="list-style-type: none"> o Within Chat Editor, choose Library ⇒ Transfer ⇒ To/From Device o Check the vocabulary file(s) (on the right side of the screen) you wish to transfer to the Editor. o Choose the To Editor button o The file name(s) will be listed in the Editor Files once complete. Choose Done. o Safely remove device from computer. Unplug USB cord from Device. <p>To view the file(s) in Chat Editor, choose Library ⇒ My Resources ⇒ Choose the Vocabulary ⇒ Open</p>
Transferring Files From Chat Editor To a Device	<ul style="list-style-type: none"> o Within Chat Editor, choose Library ⇒ Transfer ⇒ Vocabs To/From Device o Check the vocabulary file(s) (on the left side of the screen) you wish to transfer to the device. o Choose the To Device button o The file name(s) will be listed in the Editor Files once complete. Choose Done. o Safely remove device from computer. Unplug USB cord from Device. <p>To view the file(s) on the device, choose Menu ⇒ Library ⇒ Choose the Vocabulary ⇒ Open</p>
Notes about Chat Editor :	
News-2-You and Unique Pages	
Download News-2-You Pages	<ul style="list-style-type: none"> o Go to to http://saltillo.com/curriculum-partners and choose News-2-You, then select NOVA Chat/Chat Fusion o Click on the file you want to download. Then click the location size grid you want for that file. o Download and save the file somewhere you can easily find it on your computer (e.g. the desktop).
Download Unique Pages	<ul style="list-style-type: none"> o Go to to http://saltillo.com/curriculum-partners and choose Unique, then select the grade band. o Click on the file you want to download. o Save the file somewhere you can easily find it on your computer (e.g. the desktop).
Importing News-2-You and Unique Pages into Chat Editor	<ul style="list-style-type: none"> o Open the Chat Editor Import folder (should be on your desktop). o Drag and drop the downloaded file (News-2-You or Unique) into the Chat Editor Import folder. Once the file is done, close the folder. o Open Chat Editor. A pop-up box should appear that says "Importing" <ul style="list-style-type: none"> o It may take a few seconds for the box to appear. o Once it goes away the files are now in the My Resources section of the Library in Chat Editor. o Follow the steps previously described to Import the pages into the vocabulary file and create a link. <ul style="list-style-type: none"> o Make sure to import ALL the pages for the N-2-Y or Unique lesson.

Notes about **News-2-You** and **Unique**:

SALTILLO RESOURCES

Join our **Newsletter**

<http://saltillo.com/community>

Attend a **Webinar**

<http://saltillo.com/training>

Watch Client and How To **Videos**

<http://youtube.com/saltillocorp>

Learn about **Realize Language & Data Logging**

www.realizelanguage.com