WordPower: Discover Bilingual Options

Nancy Inman, M.A.T., CCC-SLP
Inman Innovations

Lisa Timm, M.S., CCC-SLP
PRC-Salttillo

© 2021 PRC-Salttillo. All Rights Reserved.

1

Housekeeping

- *Having trouble hearing or connecting to this class?*
- FIRST
 - ✓ log out and back into the training
 - ✓ re-start your computer
- If that doesn't help, call LogMeIn: 1-888-259-8414
- Training materials found in handouts section of the toolbar
- Feedback survey will pop up as you leave the class
- Type your questions in the chat window and they will be addressed by the presenter at the end, as time allows.

© 2021 PRC-Salttillo. All Rights Reserved.

2

ASHA CEUs: *If you use the ASHA CEU registry to track your CEUs*

- You must stay online on a computer the entire time and actively participate
- Submit ASHA participation form within 15 days- **Please include training date on the form**
- Instructions and participation forms are in the handouts section of the Go To Training toolbar
- Make sure to put the **name of the class and the date in the subject line of your email** when submitting the CEU form to info@prc-salttillo.com

If you don't use the ASHA CEU registry to track your CEUs you can use the attached certificate as proof of attendance.

ASHA CE
APPROVED PROVIDER

PRC-Salttillo
Introductory Level
.10 ASHA CEUs

© 2021 PRC-Salttillo. All Rights Reserved.

3

Presenter Disclosures

- Nancy Inman - owner of Inman Innovations, Inc. She receives royalties for WordPower software
- Lisa Timm - employee-owner at PRC-Salttillo

© 2021 PRC-Salttillo. All Rights Reserved.

4

Objectives

- List WordPower files available in Spanish
- List WordPower files available in French.
- Describe how the files offer bilingual support options.

© 2021 PRC-Saltito. All Rights Reserved.

5

Bilingual Options

Español

français

Profiles

Resources

© 2021 PRC-Saltito. All Rights Reserved.

6

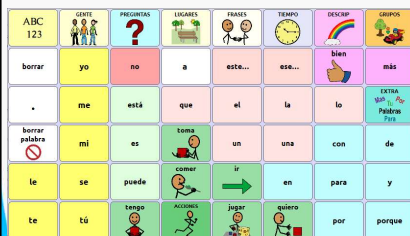
Bilingual Options

Español


© 2021 PRC-Saltito. All Rights Reserved.

7


WordPower Español

WordPower48 Español

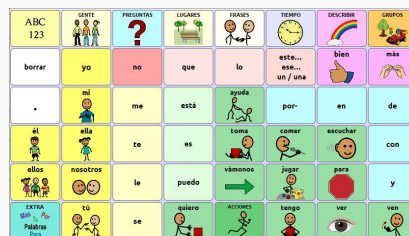WordPower48 Español Básico

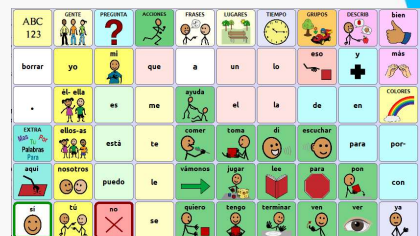WordPower60 Español Básico

© 2021 PRC-Saltito. All Rights Reserved.

8

WordPower60 Español Básico

-designed to compliment and mirror WordPower60 Basic for the bi-lingual user

WP60 Basic English

English

I like...
I don't like ...
I want + ... () or infinitive verbs)
I want to watch with you.
I am ready to ...
I love to read ...
I am watching TV.
I am going to listen to music.
What do you like to play?
I am really hungry.
I feel good.
Good job.
I don't know. / There is no...
I have a problem.
in → front / back
with you
my turn
your turn

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15

WordPower60 français

-designed to compliment
and mirror WordPower60
Basic for the bi-lingual user

WordPower60 Basic


WordPower60 français

ABC 123	PEOPLE 	QUESTION 	ACTIONS 	SOCIAL 	PLACES 	TIME 	GROUPS 	DESCRIBE 	good
clear	I	me	to	come	that	a-	the	and	more
.	my	is	eat	drink	finish	get	all	at	COLORS
EXTRA et ça la mots en haut	it	can	go	help	open	put	in	for	on
here	you	do	like	play	read	stop	out	up	off
yes	your	no	want	take	tell	turn	watch	down	with

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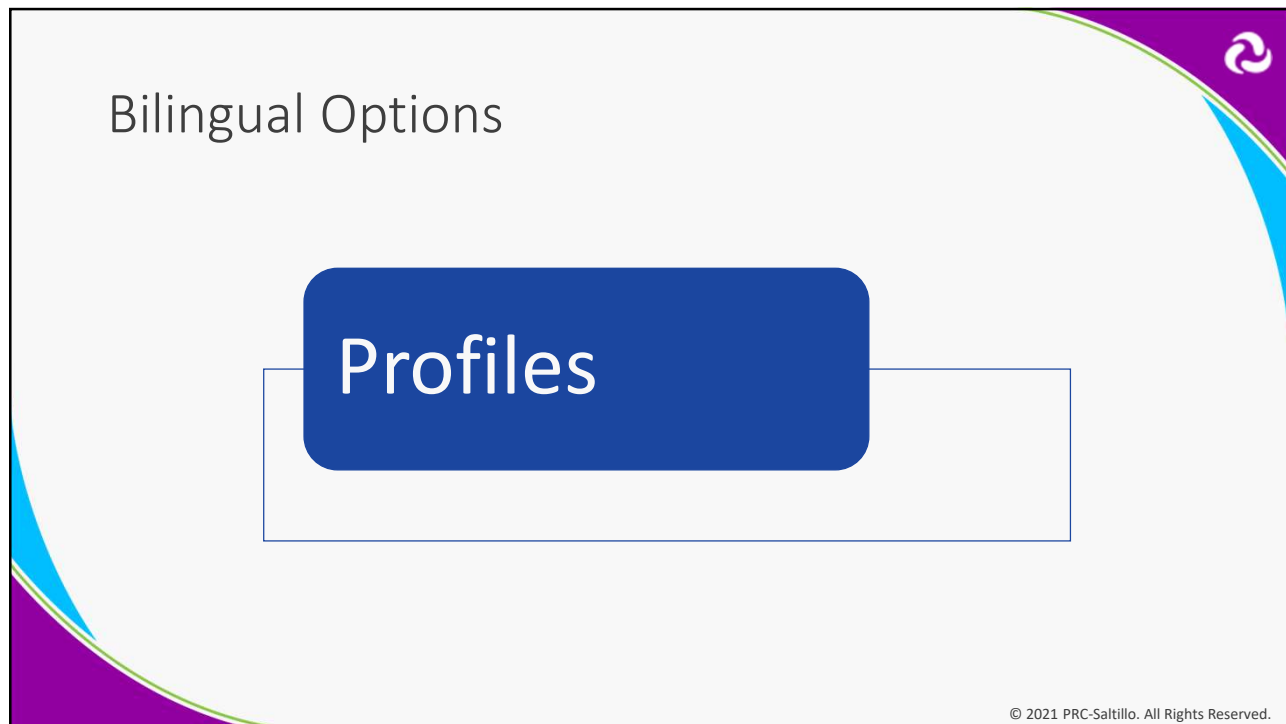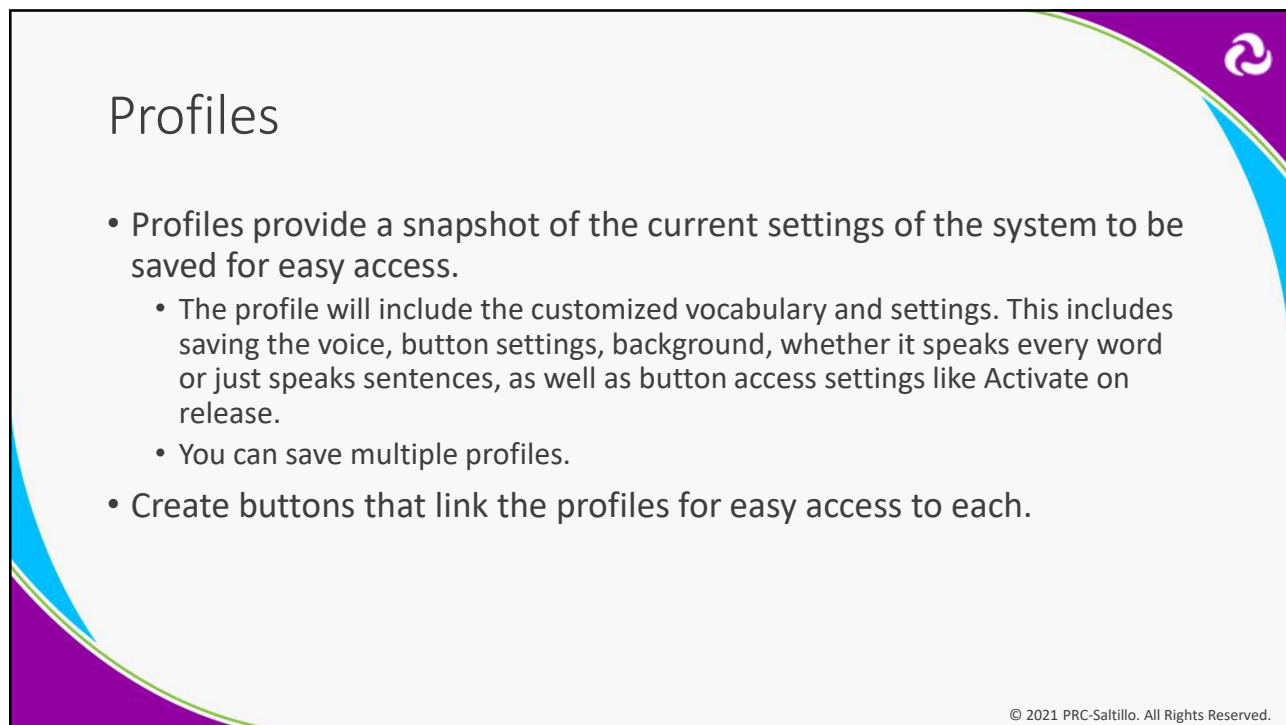
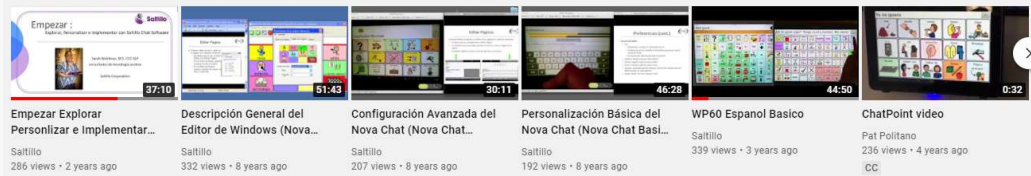
Classes/YouTube

- Saltillo [YouTube](#)

Product Features & How-To's: Español ▶ PLAY ALL

- TouchChat [YouTube](#)

TouchChat Features & How-To's: Español ▶ PLAY ALL

© 2021 PRC-Salttillo. All Rights Reserved.

21

Questions?

Nancy Inman, M.A.T., CCC-SLP
Inman Innovations

Lisa Timm, M.S., CCC-SLP
PRC-Salttillo

© 2021 PRC-Salttillo. All Rights Reserved.

22

Thank You!

- Reminder:
- Submit ASHA forms within 15 days and include today's date and the name of the class in the subject line of your email to: info@prc-salttillo.com