



1

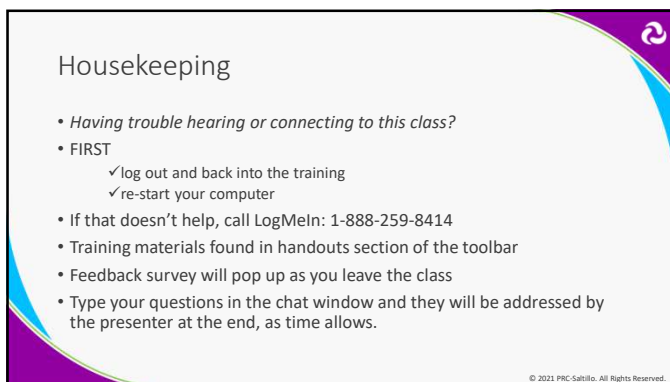

2

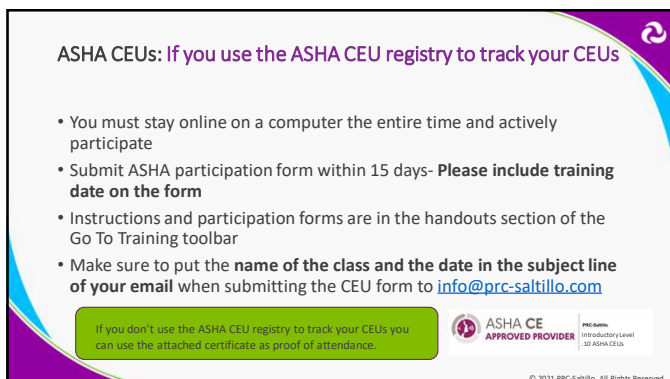

3

Presenter Disclosures

- Nancy Inman - owner of Inman Innovations, Inc. She receives royalties for WordPower software
- Lisa Timm - employee-owner at PRC-Salttillo

© 2021 PRC-Salttillo. All Rights Reserved

4




Objectives

- Describe the guiding principles used to develop WordPower vocabulary
- List three architecture features found in WordPower.
- Identify color coding used to organize WordPower.

© 2021 PRC-Salttillo. All Rights Reserved

5

WordPower found across the systems

- TouchChat App 
- NovaChat device 
- Accent with NuVoice 



© 2021 PRC-Salttillo. All Rights Reserved

6

WordPower for NovaChat & TouchChat

- 11 English
- 3 Español
- 2 French

Images:

- SymbolStyx (default)
- PCS (additional fee)

11 English WordPower Files

WordPower20 Simply, WordPower25 Touch & Scan, WordPower42, WordPower42-Basic, WordPower48, WordPower60, WordPower60-Basic, WordPower80, WordPower100, WordPower100 w/ Keyboard, WordPower140 Scan & Touch, WordPower120 Français, WordPower60 Français

3 Español WordPower Files

WordPower48- Español, WordPower48- Español Básico, WordPower60- Español Básico

NEW Français

NovaChat by Salttillo
TouchChat

© 2021 PRC-Salttillo. All Rights Reserved.

7

WordPower for Accent- NuVoice

- 17 files
- Images: Minspeak

144-WordPower, 144-Scanning WordPower, 84-WordPower w/ Keyboard, 84-Picture WordPower w/ Keyboard, 80-Scanning WordPower, 80-WordPower, 80-Picture WordPower, 40-WordPower, 60-Picture WordPower, 60-Picture WordPower Enhanced, 60-WordPower w/ Keyboard, 60-Picture WordPower Phrases and Categories, 40-WordPower, 42-Picture WordPower, 42-Picture WordPower Phrases and Categories, 30-Picture WordPower

Accent

© 2021 PRC-Salttillo. All Rights Reserved.

8

Architecture Overview

- Research Based
- Guiding Principles
- Key Elements
- Color Coding
- Basic File Characteristics
- Access: Scanning, Head Pointing

© 2021 PRC-Salttillo. All Rights Reserved.

9


WordPower – Research Based

- 500 Most Frequently Occurring Words Produced by Five Adult Communication Augmentation System Users - Beukelman, Yorkston, Poblete, Naranjo, Journal of Speech and Hearing Disorders-49 (1984)
- Core Vocabulary Determination for Toddlers, Augmentative and Alternative Communication - Banajee, M., DiCarlo, C., & Buras-Stricklin, S. (2003)
- Preschoolers' Vocabulary Arranged Alphabetically- Marvin, C. A., Beukelman, D. R., & Bilyeu, D. (1994)

© 2021 PRC-Salttillo. All Rights Reserved

10

Guiding Principles

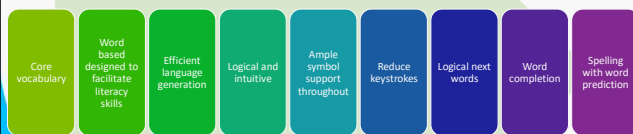

© 2021 PRC-Salttillo. All Rights Reserved

11

Key Elements of WordPower


Overview

- Core vocabulary – words arranged by grammatical type with color coding
- Robust fringe vocabulary arranged by category
- Consistency in setup and design between all vocabulary options
- Logical next words and word completion
- Literacy supports – spelling, phonics, early books, writing
- Smart punctuation (. ? !)
- Core vocabulary available within category pages
 - NovaChat/TouchChat: WP42, 60 Basic, 48/60 Español Básico
 - Accent: WP 36, 60 Enhanced
- Automatic grammatical forms:
 - I am verbing
 - I want to verb
 - I like verbing
 - I have past participle verb
- Negative contractions in subject-verb sentences
 - I can + -n't
 - I would + -n't
 - I will + -(won't)
- Grammatical morphemes available in context:
 - -ed, -ing, -s, -er, -est, -ly
- NovaChat/TouchChat: Multi-lingual – English, Español, Français

prc-salttillo

© 2021 PRC-Salttillo. All Rights Reserved


12

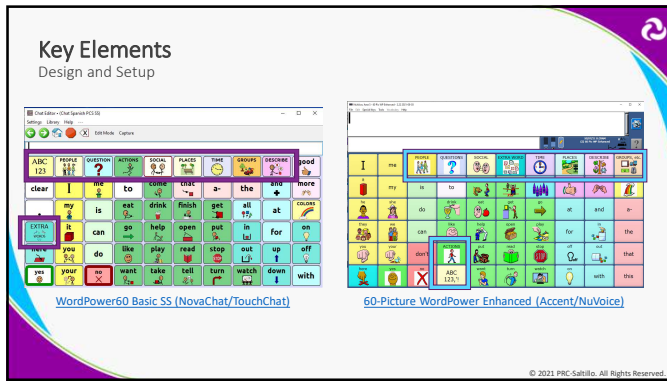

13



14

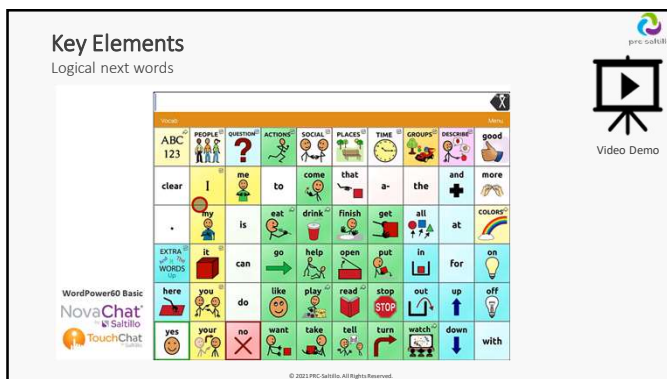

15

Key Elements

Word Completion-accurate grammar and morphology

- my -self
- your -self
- a -bout
- with -out
- up -stairs
- down-stairs
- in -side
- out -side
- help -ing
- help -s
- help -ed
- that -s
- big -er (bigger)
- big -est (biggest)

prc-salttillo
© 2021 PRC-Salttillo. All Rights Reserved.

16

Key Elements

Word Completion-accurate grammar and morphology

Accent 60-PictureWordPower Enhanced

Video Demo

© 2021 PRC-Salttillo. All Rights Reserved.

17

Color coding:

Fitzgerald Key

- Pronouns/PEOPLE
- Verbs/ACTION Words
- Adjectives/DESCRIBE
- QUESTIONS
- Nouns/GROUPS

WordPower Wednesday

© 2021 PRC-Salttillo. All Rights Reserved.

18

WordPower Basic/Picture WordPower Files

NovaChat/TouchChat

- *English*
 - WordPower42 Basic
 - WordPower60 Basic
- *Spanish*
 - WordPower48 Espanol Basico
 - WordPower60 Espanol Basico
- *Simple and Phrase*
 - WordPower20 Simply
 - WordPower25 Touch & Scan

Accent-NuVoice

- 60-Picture WordPower Enhanced
- 60-Picture WordPower Phrases & Categories
- 45-Picture WordPower
- 45-Picture WordPower Phrases & Categories
- 36-Picture WordPower
- 28-Picture WordPower



19

The Basic Files/Picture WordPower

1. **** Increased symbol support**
2. Yes/No on home and category pages (WP60 & WP60 Español Básico, 60PWP Enhanced)
3. Sentence starters
4. Symbols chosen for easy identification and transparency
5. **** Several high frequency core verbs link directly to category pages**
6. Early concepts with picture support
7. Daily routines
8. Lock Page function
9. Social pages for quick interaction and emotional regulation
10. Early books to support literacy and engagement
11. Low-tech boards

© 2021 PRC-Salttillo. All Rights Reserved.

20

How the Basic Files/Picture WordPower Enhanced navigate...

Random Skin Tone Setting

WordPower60 NovaChat Salttillo TouchChat

Files demonstrated

- 1) WP60 vs WP60 Basic
- 2) 60-PWP vs 60 PWP Enhanced

© 2021 PRC-Salttillo. All Rights Reserved.

21

WordPower Basic Files/Picture WordPower

Core vocabulary embedded within category pages to facilitate easy generation of language

FOOD page

© 2021 PRC-Salttillo. All Rights Reserved

22

WordPower Basic/Picture WordPower

Core embedded

Video Demo

WordPower60 Basic
NovaChat
TouchChat

prc-salttillo

© 2021 PRC-Salttillo. All Rights Reserved


23

Literacy Supports

Early books

© 2021 PRC-Salttillo. All Rights Reserved


24

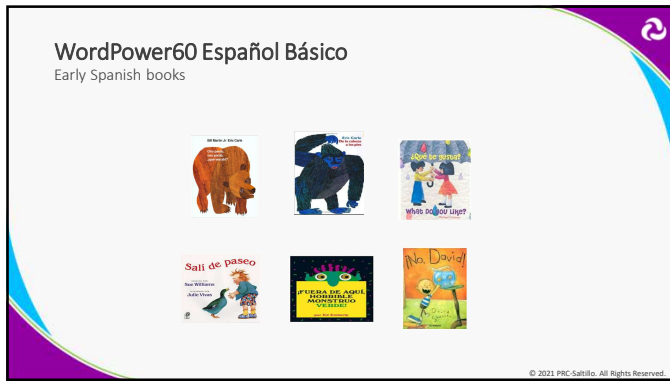

25



26

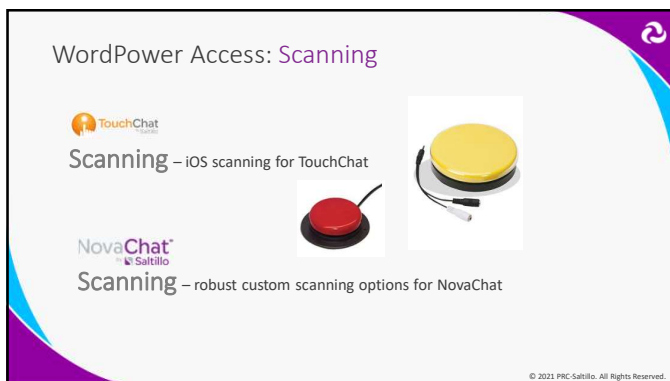

27

WordPower25 Touch & Scan

Scanning

- iOS scanning for TouchChat app
- Custom scanning for NovaChat

© 2021 PRC-Satillo. All Rights Reserved

28

WordPower140

Scanning

- Sophisticated custom scanning on NovaChat

***Predictive reduction

© 2021 PRC-Satillo. All Rights Reserved

29

Head Tracking for TouchChat app with iPad Pro

iOS facial recognition feature

- Enable Head Tracking within TouchChat
- Compatible with:
 - iPhone
 - iPhone 12 Pro Max
 - iPhone 12 Pro
 - iPhone 12 mini
 - iPhone 12
 - iPhone 11 Pro Max
 - iPhone 11 Pro
 - iPhone 11
 - iPhone XS Max
 - iPhone XS
 - iPhone XR
 - iPhone X
 - iPad
 - iPad Pro 12.9-inch (4th generation)
 - iPad Pro 12.9-inch (3rd generation)
 - iPad Pro 11-inch (2nd generation)
 - iPad Pro 11-inch

© 2021 PRC-Satillo. All Rights Reserved

30

144 WordPower
Scanning

I	me	my	it	they	to	-s	-'s	Device	SOCIAL	PEOPLE	TIME	really	so	very	
you	your	he	she	we	want	be-	go	like	reset	e	a	r	d	u	v
what	where	how	who	have	is	are	was	were	Share it	t	o	i	l	g	
-n't	not	when	why	good	do	can	could	did	Shift & Sym	n	s	f	y	x	
?	a	the	that	this	more	had	will	would	does		h	c	p	k	j
time	now	day	today	DESCRIBE	PLACES	GROUPS	any every one			m	b	w	q	z	
come	drink	eat	feel	finish	read	say	see	please	thank you	with	about	and	at	be-cause	but
get	give	help	listen	look	stop	take	talk	tell	here	down	for	from	if	in	of
make	need	open	play	put	turn	watch	wear	work	there	off	on	or	out	over	up

Accent

© 2021 PRC-Satillo. All Rights Reserved.

31

84 WordPower
Scanning


I	?	PEOPLE	DESCRIBE	TIME	PLACES	GROUPS	DEVICES	SOCIAL	more	really	
me	my	.	reset	e	a	r	n	s	f	y	x
it	don't	Shift & 123	d	u	g	b	h	c	p	k	j
they	Help verbs	any every some	t	o	i	l	m	v	w	q	z
we	can	do	to	be-	come	drink	eat	and	at	a-	the
you	your	did	have	get	go	help	like	for	in	of	that
was	need	play	want	work	on	out	with	this			

Accent

© 2021 PRC-Satillo. All Rights Reserved.

32

WordPower with Look/NuEye module:
Eye Tracking



Accent

© 2021 PRC-Satillo. All Rights Reserved.

33

WordPower with NuPoint: Head Tracking



© 2021 PRC-Satillo. All Rights Reserved

34

28 & 36 Picture WordPower Demo



36 Picture WordPower



28 Picture WordPower



© 2021 PRC-Satillo. All Rights Reserved

35

36 Picture WordPower Practice Sentences

- I will eat peanut butter and crackers.
- I am feeling really great.
- That is lot of work.
- You are doing a good job.
- I finished my work.
- I want to play dolls with you.
- I want to (FAVORITE THINGS) ...



© 2021 PRC-Satillo. All Rights Reserved

36

28 Picture WordPower

Practice Sentences

- I want my shoes. I want my shoes.
- I like to dance. I like to dance.
- Can I have a ball? Can I have a ball?
- I am feeling proud of you.
- I love dogs and turtles.
- I want the fire engine
- I want to (FAVORITE THINGS) ...



© 2021 PRC-Salttillo. All Rights Reserved

37

Questions?

Nancy Inman, M.A.T., CCC-SLP
Inman Innovations



Lisa Timm, M.S., CCC-SLP
PRC-Salttillo



© 2021 PRC-Salttillo. All Rights Reserved

38

Thank You!

- Reminder:
- Submit ASHA forms within 15 days and include today's date and the name of the class in the subject line of your email to: info@prc-salttillo.com



© 2021 PRC-Salttillo. All Rights Reserved

39
